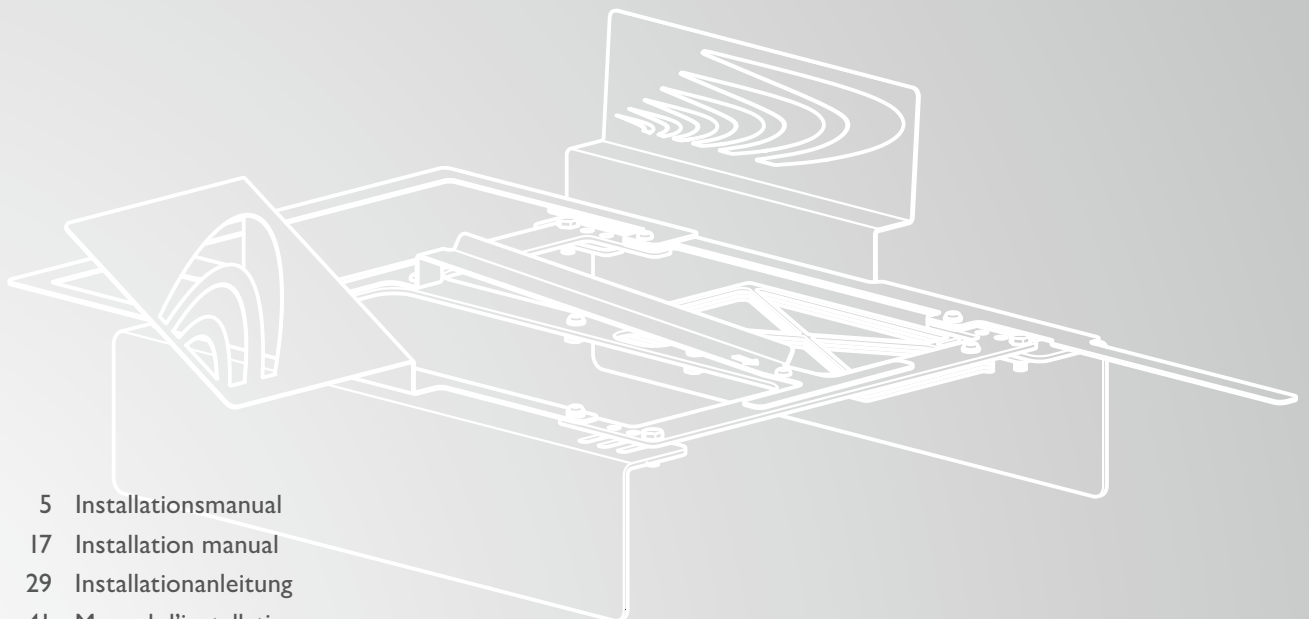
Turny Low Vehicle

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Thank you for choosing a

Universal bracket from Autoadapt!

The manual that follows is an integral, important part of the product, providing you with information on how to perform accurate and safe installation of the product. Please read through the manual carefully before starting your work.

If you have any questions about your customer's adaptation, please feel free to contact Autoadapt. Your feedback is important to us. It helps us in our work of continuously improving our product and its installation. You can find more information about us and our range of car adaptation solutions by visiting www.autoadapt.com.

Once again, thank you for placing your confidence in our products!

Autoadapt

Safe vehicle adaptation solutions

For your safety Autoadapt products are designed and tested according to current directives and standards.

The application tool's function

- 18 Right model
- 20 Left model

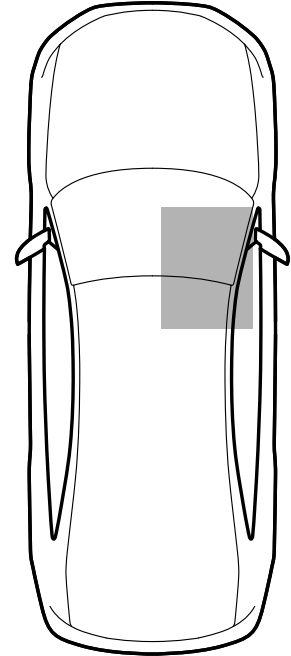
Adjusting the brackets

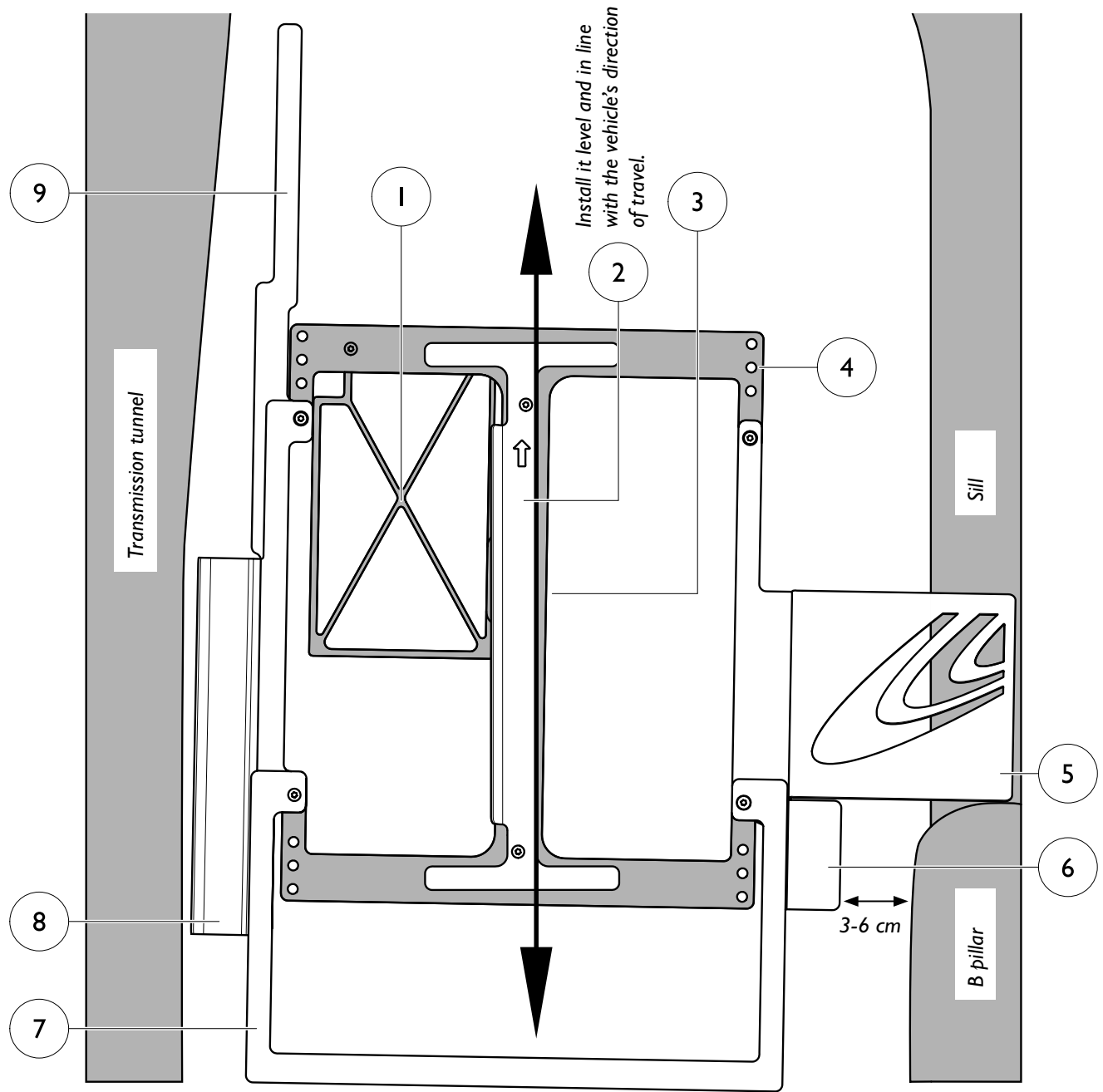
- 22 Box contents
- 23 Position the application tool in the vehicle
- 25 Adjusting the brackets
- 26 Adapt the console leg
- 27 Install the bracket in the vehicle

The application tool's function

Right model


1. The grey-marked part of the application tool corresponds to the base plate of Turny Low Vehicle and the cross frame corresponds to the unit's crash segment. The cross frame must not rest on the bracket and must instead be positioned to the side for the installation to be level and stable.
2. Ensure the correct incline (4°) and optimal offset (1°) of Turny Low Vehicle. Install it level and in line with the vehicle's direction of travel.
3. Hole in the base plate for wiring that runs from Turny Low Vehicle (not shown in figure).
4. Three attachment points in each corner for mounting Turny Low Vehicle on the side bracket.
5. If possible, should abut the B pillar and sill when adjusting the side bracket.
6. Shows Turny Low Vehicle's relationship to the B pillar. When programming, a short distance to the B pillar results in reduced legroom, but greater seat reach outside the vehicle. A greater distance has the opposite effect, with more legroom but reduced seat reach outside the vehicle.
7. Shows how much space is needed to move Turny Low Vehicle back in order to access the forward attachment points when mounting on the side bracket. Note that Turny Low Vehicle can be moved a further 10 cm back.
8. Shows Turny Low Vehicle's relationship to the transmission tunnel.
9. Shows Turny Low Vehicle's programmable forward end position.


Left model

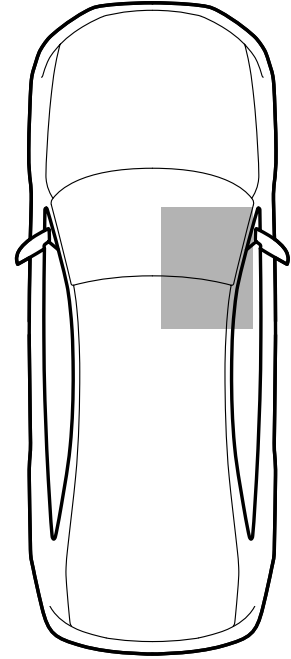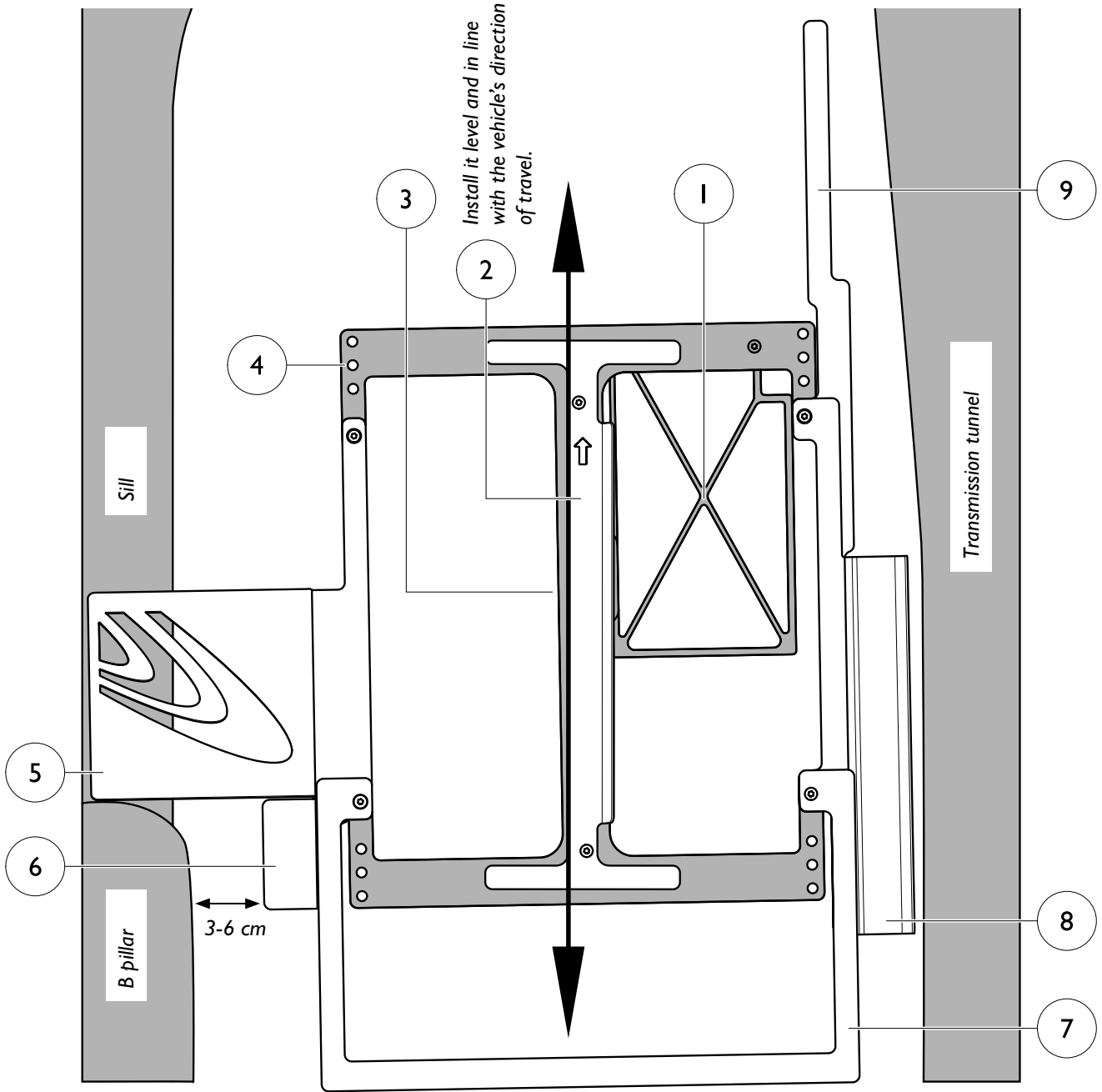1. The grey-marked part of the application tool corresponds to the base plate of Turny Low Vehicle and the cross frame corresponds to the unit's crash segment. The cross frame must not rest on the bracket and must instead be positioned to the side for the installation to be level and stable.
2. Ensure the correct incline (4°) and optimal offset (1°) of Turny Low Vehicle. Install it level and in line with the vehicle's direction of travel.
3. Hole in the base plate for wiring that runs from Turny Low Vehicle (not shown in figure).
4. Three attachment points in each corner for mounting Turny Low Vehicle on the side bracket.
5. If possible, should abut the B pillar and sill when adjusting the side bracket.
6. Shows Turny Low Vehicle's relationship to the B pillar. When programming, a short distance to the B pillar results in reduced legroom, but greater seat reach outside the vehicle. A greater distance has the opposite effect, with more legroom but reduced seat reach outside the vehicle.
7. Shows how much space is needed to move Turny Low Vehicle back in order to access the forward attachment points when mounting on the side bracket. Note that Turny Low Vehicle can be moved a further 10 cm back.
8. Shows Turny Low Vehicle's relationship to the transmission tunnel.
9. Shows Turny Low Vehicle's programmable forward end position.

Adjusting the brackets

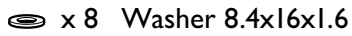
Box contents


Under no circumstances is it permitted to weld in the construction. The console is only crash-tested and approved based on the installation procedure which are described in this manual.


x 20 Screw M8x20


x 8 Washer 8.4x16x1.6

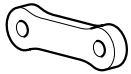
x 4 Two-hole bolts

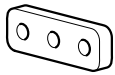x 4 Three-hole bolts

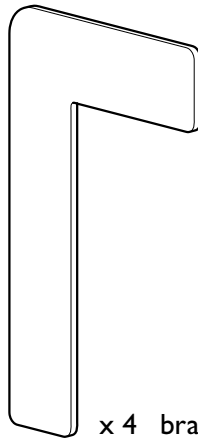x 4 bracket leg

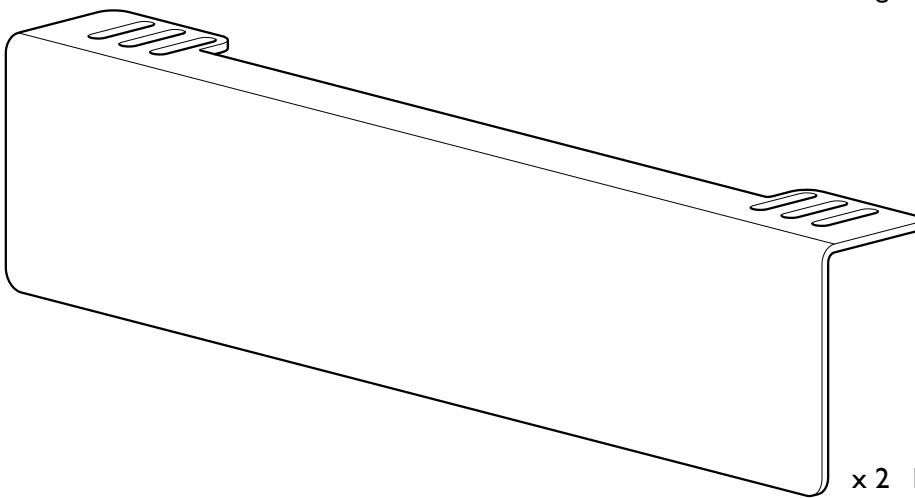
x 2 L profile bracket


(Only referred to as bracket in the following pages)

Position the application tool in the vehicle


- | All illustrations in this chapter show installation of a Turny Low Vehicle that swings right. The procedure for units that swing left is the same. Begin by fitting the brackets to the application tool. Fasten the screws loosely to more easily adjust bracket location in the vehicle.

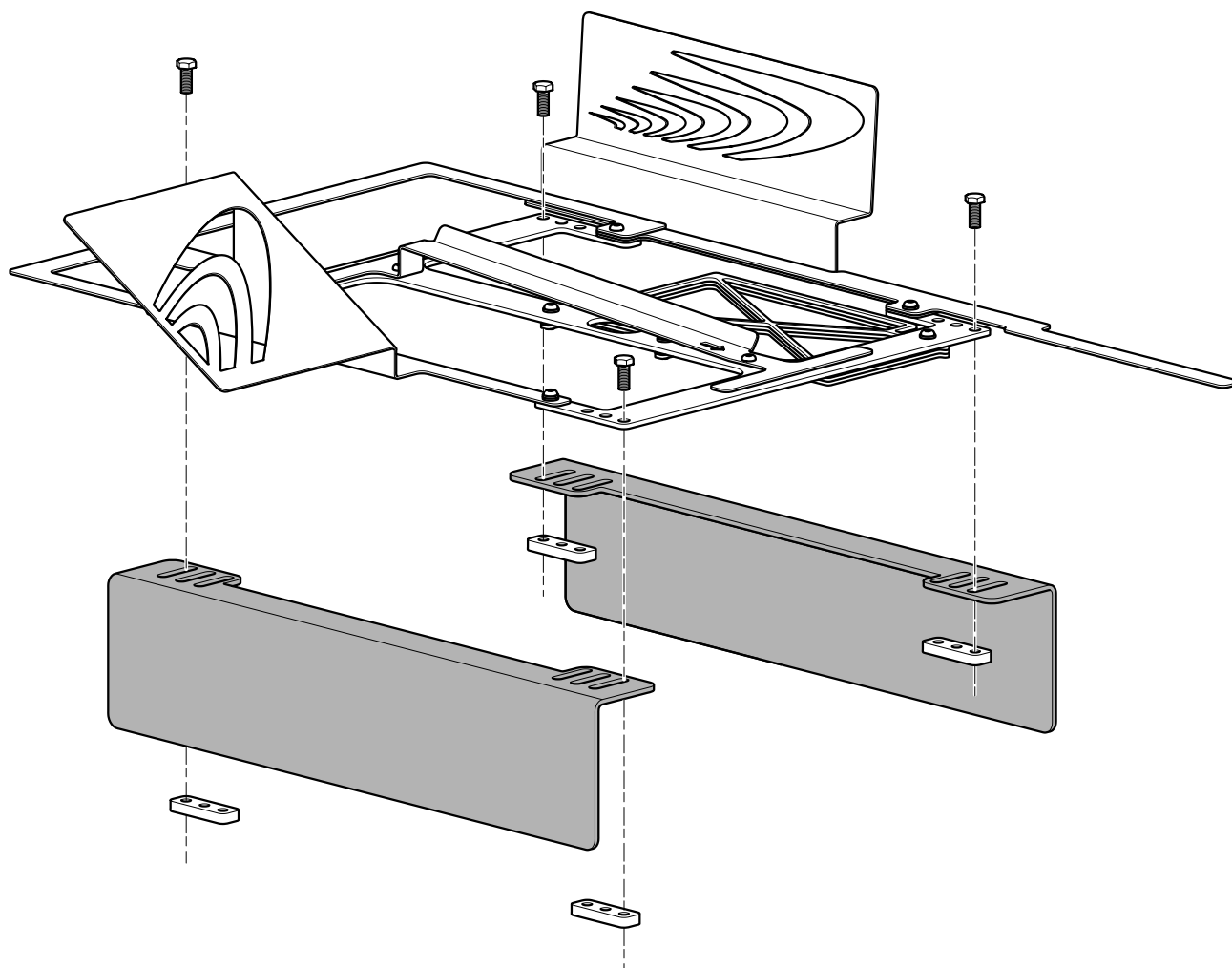- Place the application tool in the vehicle and align the part marked HORIZONTAL AREA with the vehicle's direction of travel. This will ensure an optimal offset of 1 degree. The recommended offset is 0-2 degrees. Next, adjust the brackets laterally. The distance between bracket and the centre of the OEM seat attachment must be between 15-45 mm for installation to be stable. A distance of at least 15 mm will also ensure that the head of the screw will not collide with the curvature of the bent leg when installing. With the aid of the application tool, make sure there is sufficient space for the Turny Low Vehicle movement paths. If necessary, the bracket closest to the door can be inverted laterally to achieve an optimal position. The bracket closest to the transmission tunnel may not be turned round otherwise it will end up within Turny Low Vehicle's crash segment.

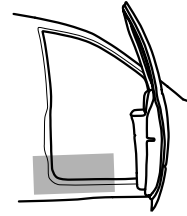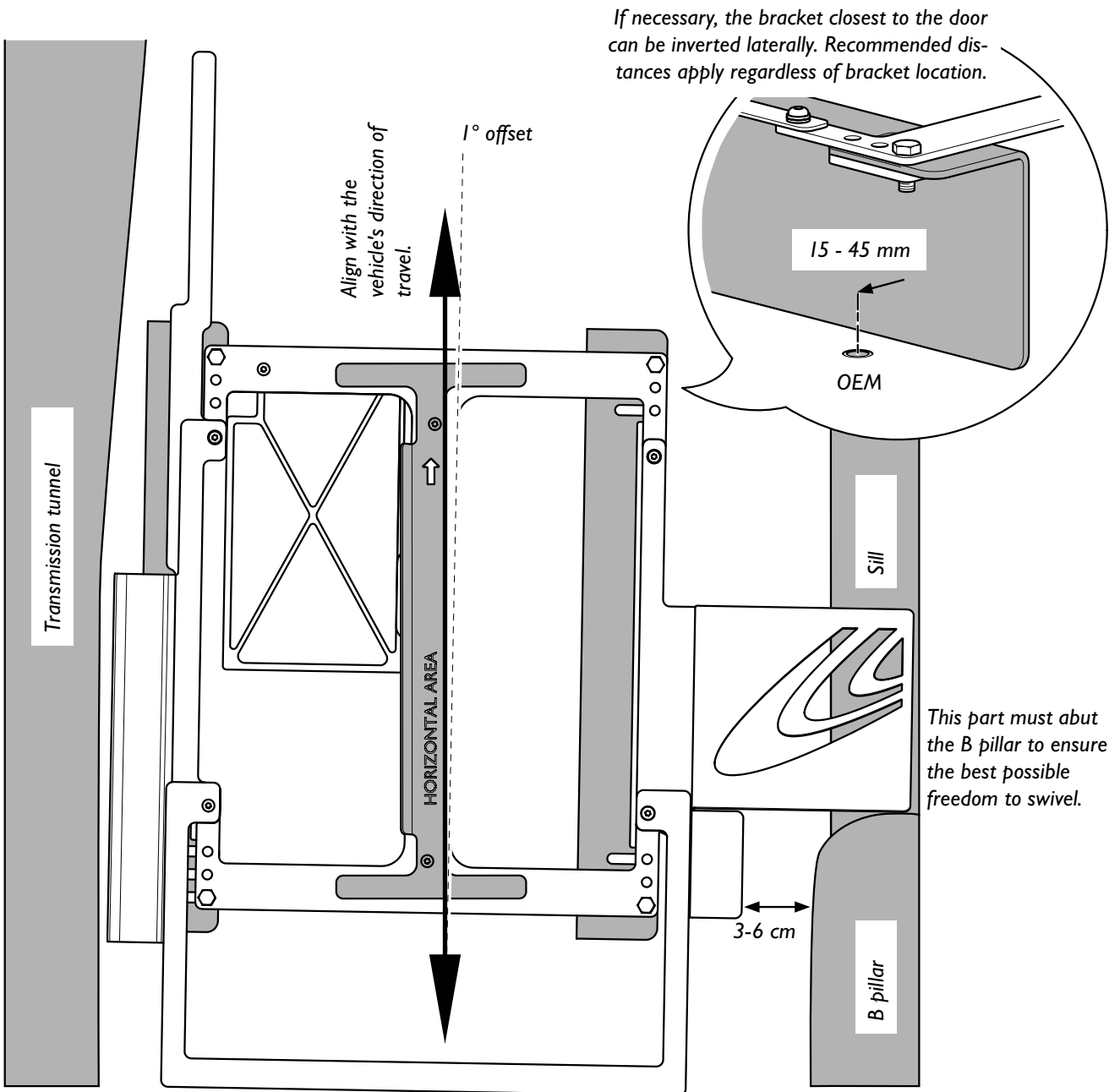As far as possible, endeavour to align the brackets as parallel as possible. This may mean that the OEM seat attachment ends up on both sides of the bracket, which is a correct alternative solution.

Adjusting the brackets

- 1 Turny Low Vehicle must be inclined and installed 4 degrees lower at the rear. When the part marked HORIZONTAL AREA is level, the incline is correct. Vehicle floors often already have this incline, in which case only bracket height need be adjusted. Begin by taking the measurements below. At the same time, determine how the bracket legs should be configured; see next page.

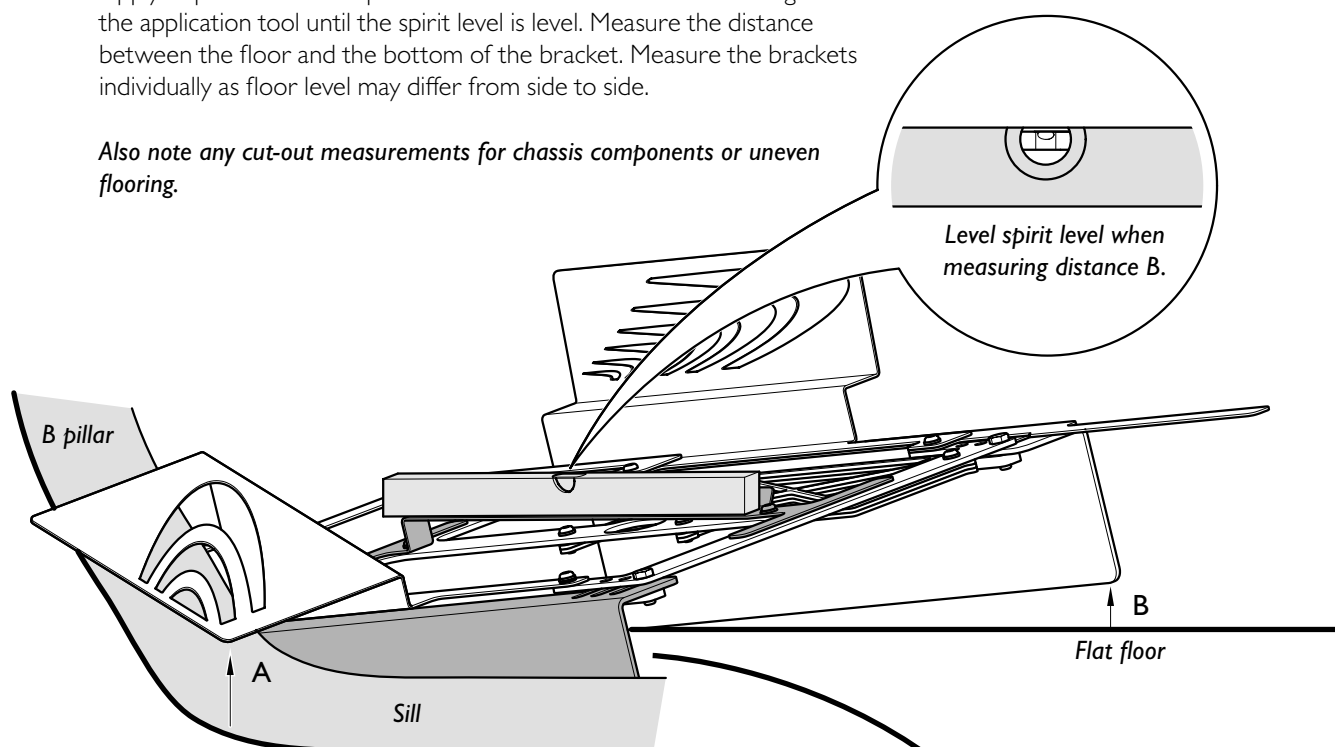
A / Measurement for adapting the height (always)

- **Do not use** a spirit level. Measure the distance between the sill and the application tool.

B / Measurement for adjusting incline (if floor incline other than 4 degrees)

- Apply a spirit level to the part marked HORIZONTAL AREA. Angle the application tool until the spirit level is level. Measure the distance between the floor and the bottom of the bracket. Measure the brackets individually as floor level may differ from side to side.

Also note any cut-out measurements for chassis components or uneven flooring.


- 2 Remove the brackets from the application tool, mark out the distances and cut. The following is an example of how to mark out distances in cases where both height and incline must be adjusted.

A = 4.5 cm

- add a further 0.5 cm to provide room for the console leg or mat.

B = 3 cm

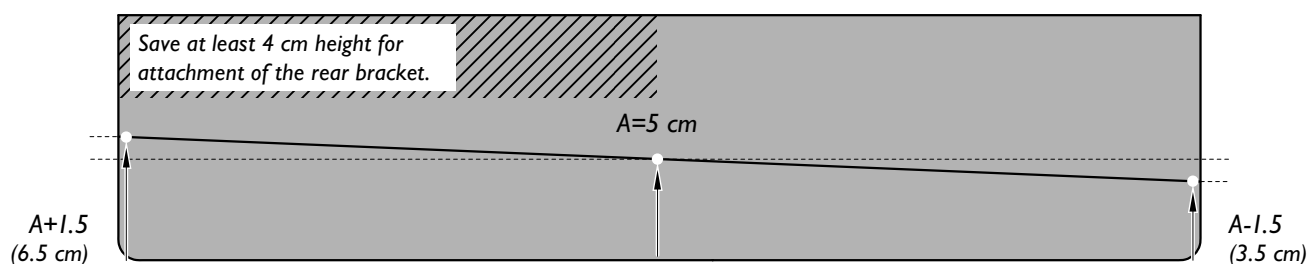- divide the measurement into two and subtract from or add to the A measurement to achieve the correct inclination.


Adapt the console leg

I Bend, twist and place the bracket legs based on the conditions that prevail in the vehicle. The short part of the leg must however always be installed on the inside or outside of the bracket. Allow the conditions in the vehicle combined with the aesthetics to determine what is best. The design is very forgiving which simplifies the adaptation since there is some adjustment range in the installing itself.

To achieve a stable construction the bracket legs are located as far apart as possible, out towards the short sides of the brackets. Begin by deciding leg location and then bend or twist it towards the OEM seat attachment. Keep in mind that the distance between the bracket and centre of OEM seat attachment must be between 15-45 mm.

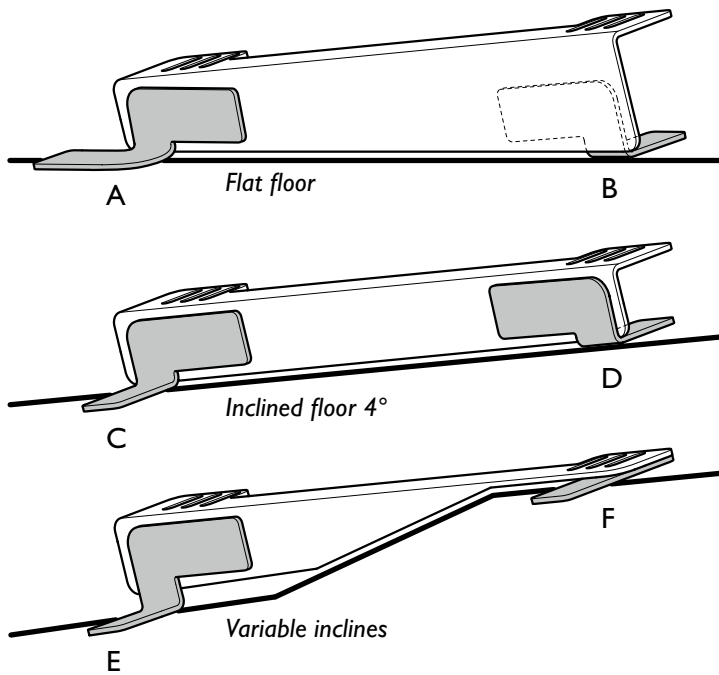Example of adjusted brackets and bracket legs.

A / A bracket leg configured to reach an attachment that is slightly to one side.

B / Configured bracket leg located on the inside of the bracket such that it is not visible.

E / Configured bracket leg located on the outside of the bracket.

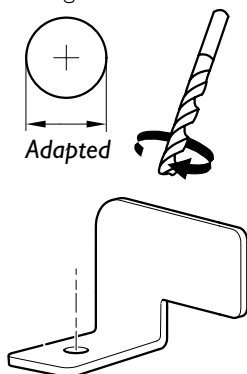
D / Configured bracket leg used as a spacer between the mat and the bracket.

C / Configured bracket leg used to adjust the height at the back of the bracket.


F / For front leg only. Located under the bracket to reduce installation height.


2 Drill holes in the dimensions which are adapted for OEM seat attachment. Centre the tool for the best strength.

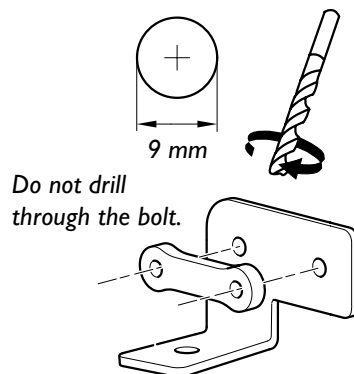Drill holes for attaching the bracket. Use the two-hole bolt as a template to mark out the hole's position.

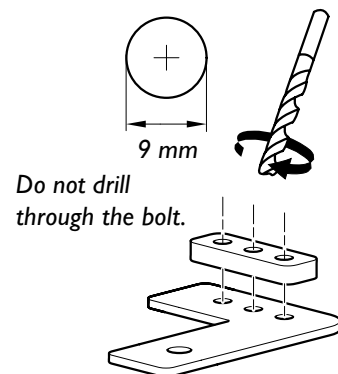If option F is practised above, use the three-hole bolt as a template to mark out the hole's position.


Install the bracket in the vehicle

- 1 Install all console legs in OEM seat attachment in the floor. Use a spacer to compensate for mat thickness.
- 2 Fit the brackets to the application tool again and put it back into the intended position in the vehicle.


Steps 2 and 3 – lay the spirit level on top of the part marked HORIZONTAL AREA as indicated by the arrows. The spirit level must be horizontal in all three positions to ensure correct installation. This is essential for problem-free product function.

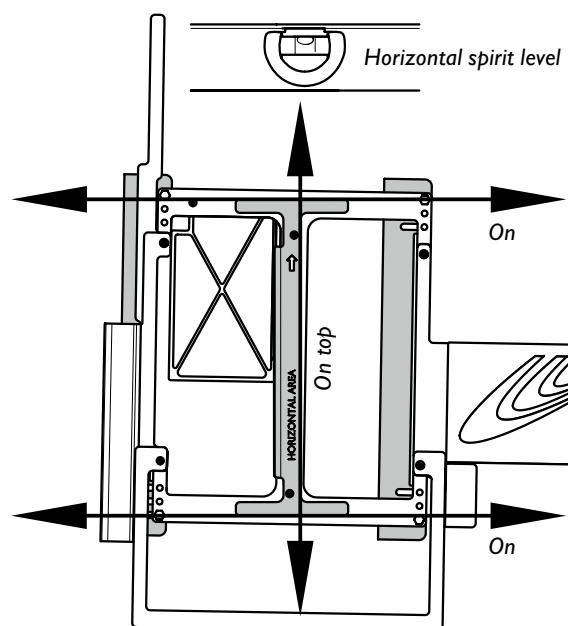Mark where the hole for each bracket leg will be drilled into the brackets. Next, remove the brackets from the application tool and drill. Use 9 mm drill.

EN

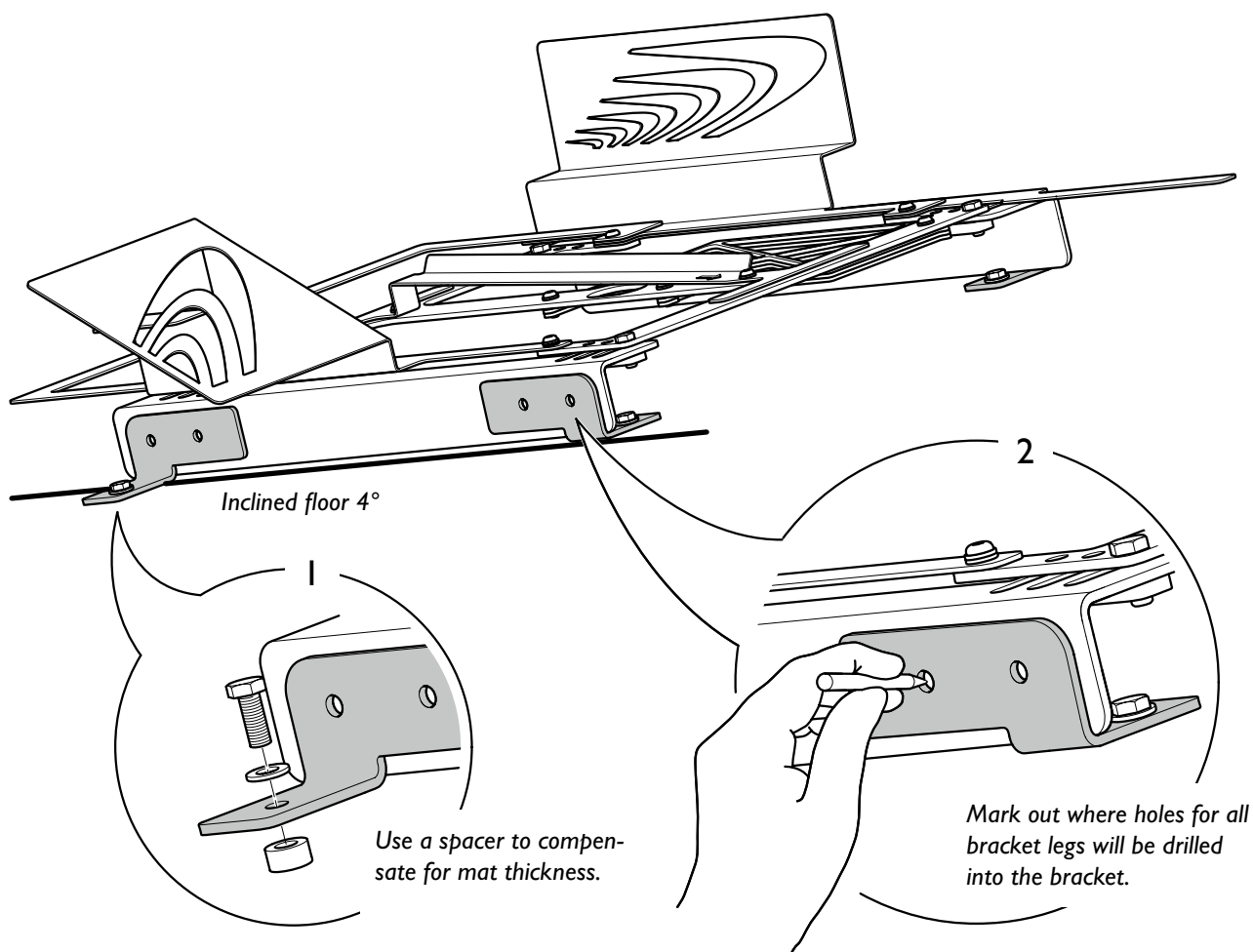Installation


Spacers between 5-45 mm height are available to buy as accessories. Contact Autoadapt Support for further help.

3 Fit the brackets to the bracket legs. Options 1 or 2 applies to all types of bent and twisted legs.

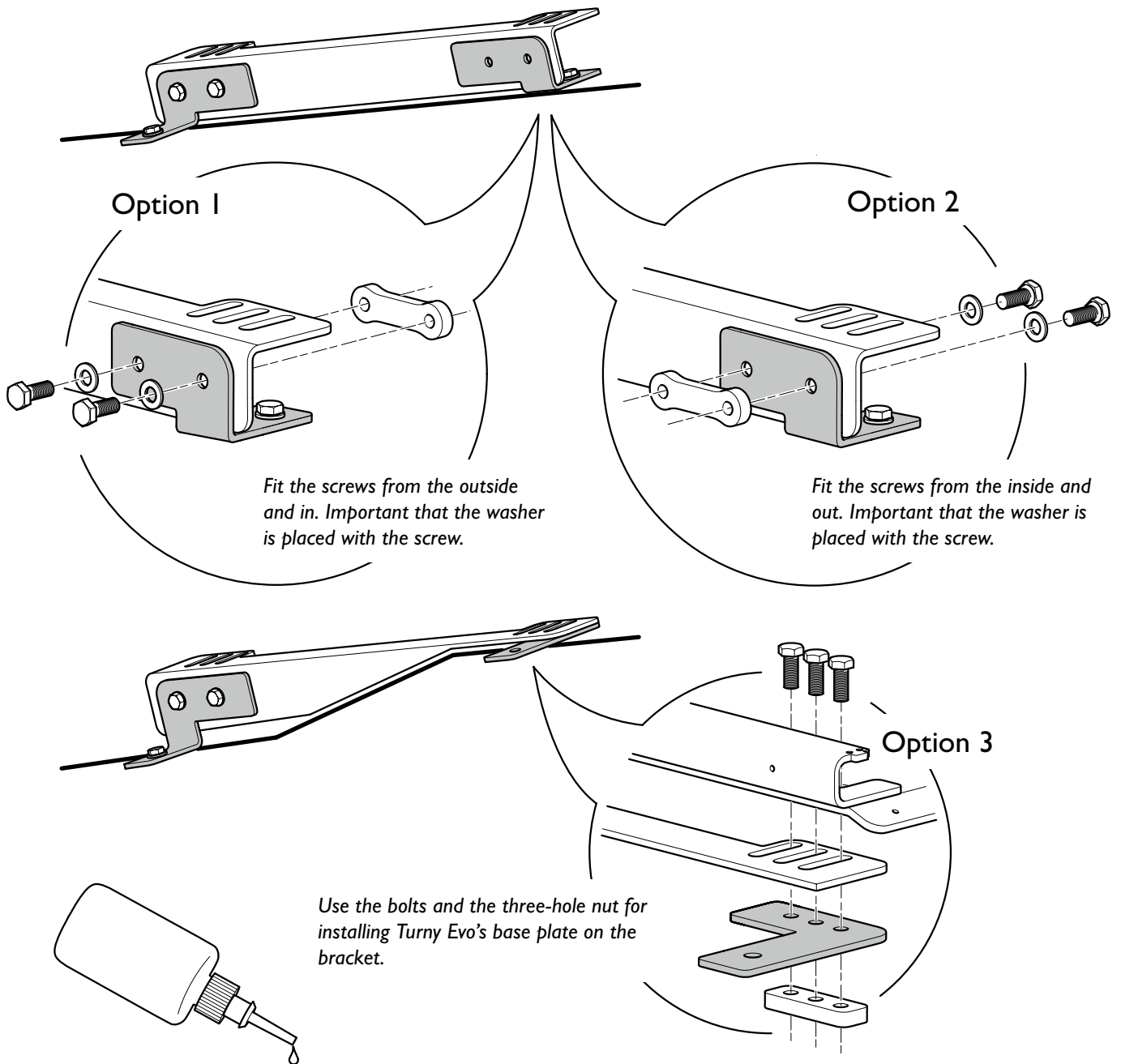Alternative 3 applies only to installation of front bracket legs as per example F in the Adapting bracket legs chapter.

Apply thread locking on all screw connections at the final installation.

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Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005
Quality system certified in accordance with ISO 9001:2008

Illustrations, descriptions and specifications in the user manual are based on current product information.
Autoadapt reserves the right to make alterations without previous notice.

© 2015 Autoadapt AB

www.autoadapt.com

facebook.com/autoadapt
twitter.com/autoadapt

AUTOADAPT AB

Åkerivägen 7
S-443 61 Stenkullen
Sweden
Phone: +46 302 254 00
E-mail: info@autoadapt.com

Worldwide Safe Mobility Solutions

Included in our corporate group are Autoadapt,
BraunAbility, Bruno, Unwin and Foca-Braun.