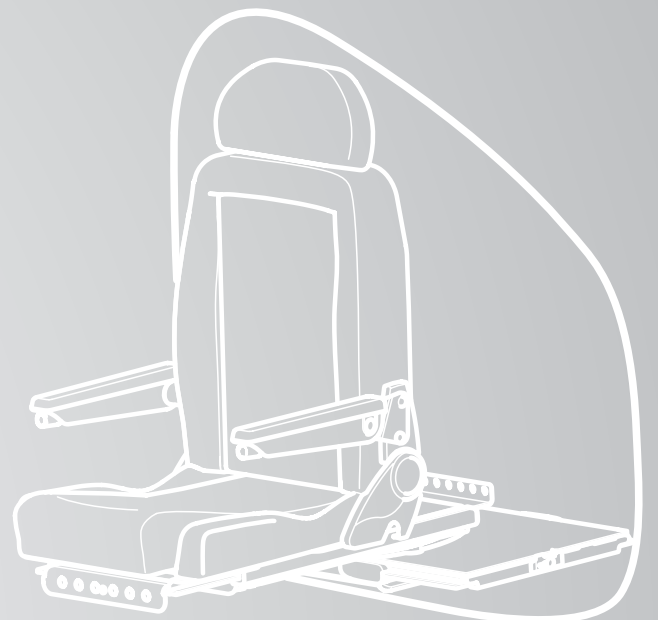
Turny Low Vehicle

Autoadapt | Swivel seat

SV	5	Installationsmanual
EN	21	Installation manual
DE	37	Installationanleitung
FR	53	Manuel d'installation
ES	69	Manual de instalación
IT	85	Manuale di installazione

SV

EN

DE

FR

ES

IT

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Thank you for choosing a

Turny Low Vehicle from Autoadapt!

The manual that follows is an integral, important part of the product, providing you with information on how to perform accurate and safe installation of the product. Please read through the manual carefully before starting your work.

If you have any questions about your customer's adaptation, please feel free to contact Autoadapt. Your feedback is important to us. It helps us in our work of continuously improving our product and its installation. You can find more information about us and our range of car adaptation solutions by visiting www.autoadapt.com.

Once again, thank you for placing your confidence in our products!

Autoadapt

Safe vehicle adaptation solutions

For your safety Autoadapt products are designed and tested according to current directives and standards.

Installation

22	Route and install cables in the car
22	Install the bracket in the car
23	Place Turny Low Vehicle on the bracket
23	Connect Turny Low Vehicle
24	Install Turny Low Vehicle on the bracket
24	Program new movement paths
25	Install mount for lower seat belt anchoring point
25	Fit seat
26	Fit seat belt
28	Install covers
30	Program Turny Low Vehicle
33	Update the software
34	Troubleshooter


Technical specification

Installation

I. Route and install cables in the car

Route and install cables for the hand-held control, door switch, earth and 12V. The illustration below shows the associated connection points in the unit.

For a complete wiring diagram, see the back of the manual.

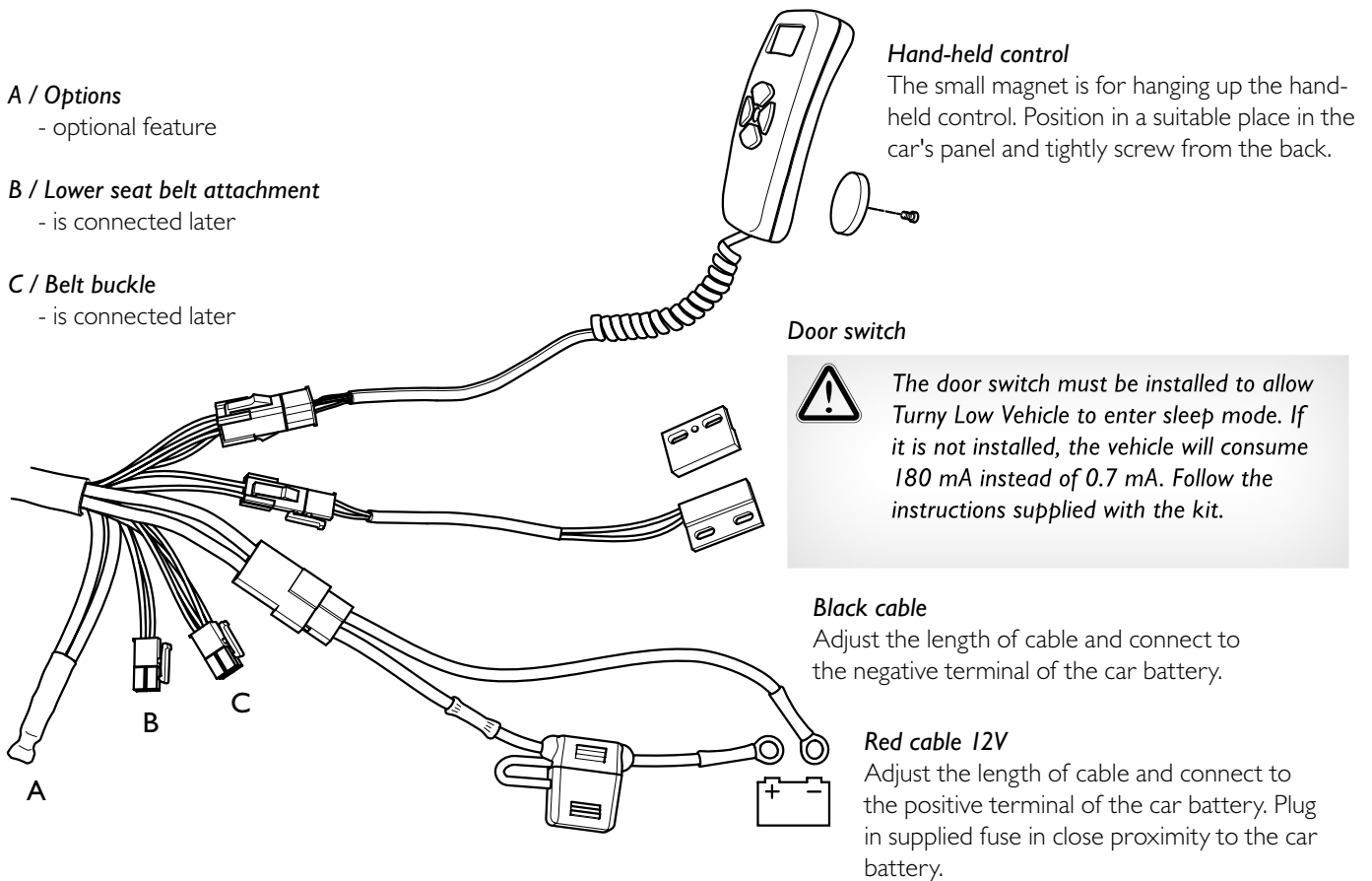2. Install the bracket in the car

A bracket is necessary between the product and the floor of the car to install Turny Low Vehicle. We recommend a car-specific model. In cases where a car-specific bracket is lacking, a universal bracket is available. For the full bracket range, see, dealerweb.autoadapt.com

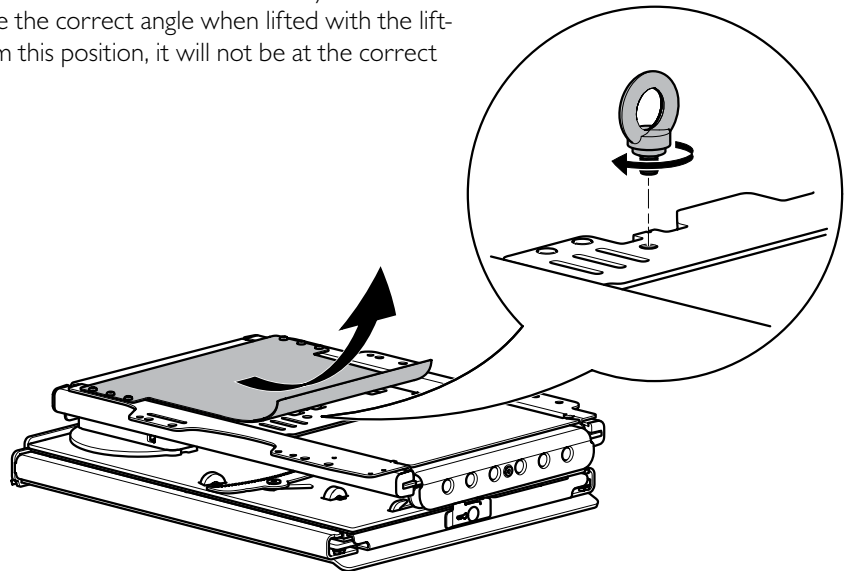
To install the bracket, follow the relevant instructions enclosed with the bracket.

3. Place Turny Low Vehicle on the bracket

Install Turny Low Vehicle inclined 4 degrees to the rear. Use the lifting eye supplied with the application tool to make positioning the unit on the bracket easier. Turny Low Vehicle is delivered positioned such that it will assume the correct angle when lifted with the lifting eye. If Turny Low Vehicle is disturbed from this position, it will not be at the correct inclination.


3.1 Screw the lifting eye in place

Lift the plastic cover to access the lifting eye attachment point. Screw the eye all the way down.

3.2 Place the unit on the bracket

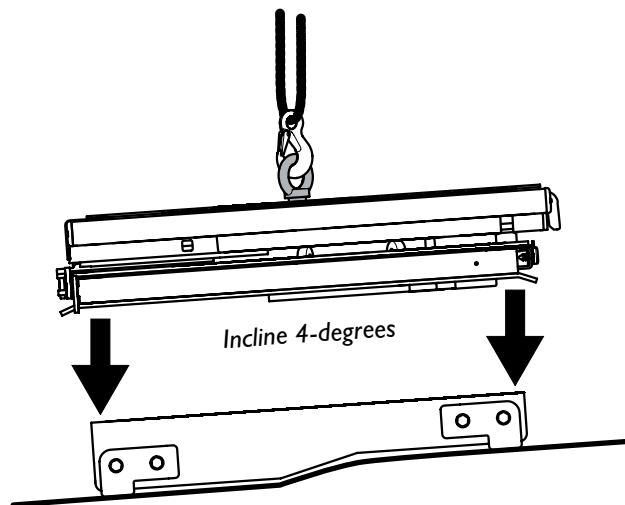Turny Low Vehicle is delivered positioned such that the unit is at the correct angle when lifted with the lifting eye. Place the unit on the bracket and then remove the lifting eye.



4. Connect Turny Low Vehicle

Connect the hand-held control cabling, door switch and power to the unit. Upon delivery, Turny Low Vehicle is preprogrammed for operation as soon as the hand-held control is connected. Make sure voltage is above 10.5V. The equipment requires a voltage above 10.5V in all modes.

Manoeuvre carefully. Before Turny Low Vehicle is programmed with a suitable path, there is a risk of damage to the unit and the vehicle.

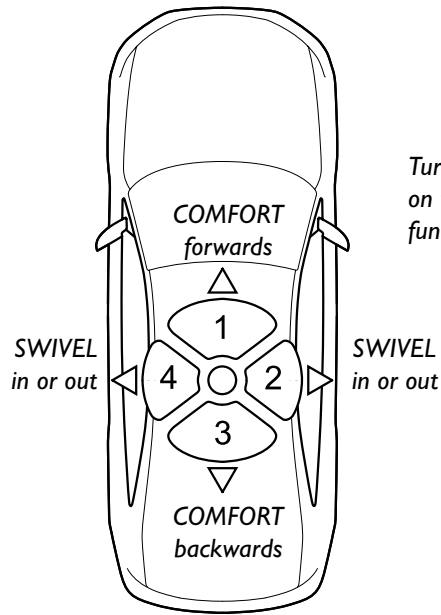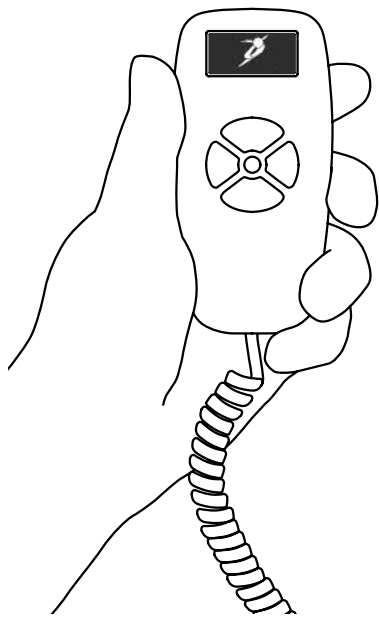
Optional function buttons light up green. Hold the selected button down. When the seat reaches its preprogrammed end position, it will stop automatically. To stop movement, release the button.

Turny Low Vehicle's location in the vehicle on the right or left side determines the functions of buttons 2 and 4.

5. Install Turny Low Vehicle on the bracket

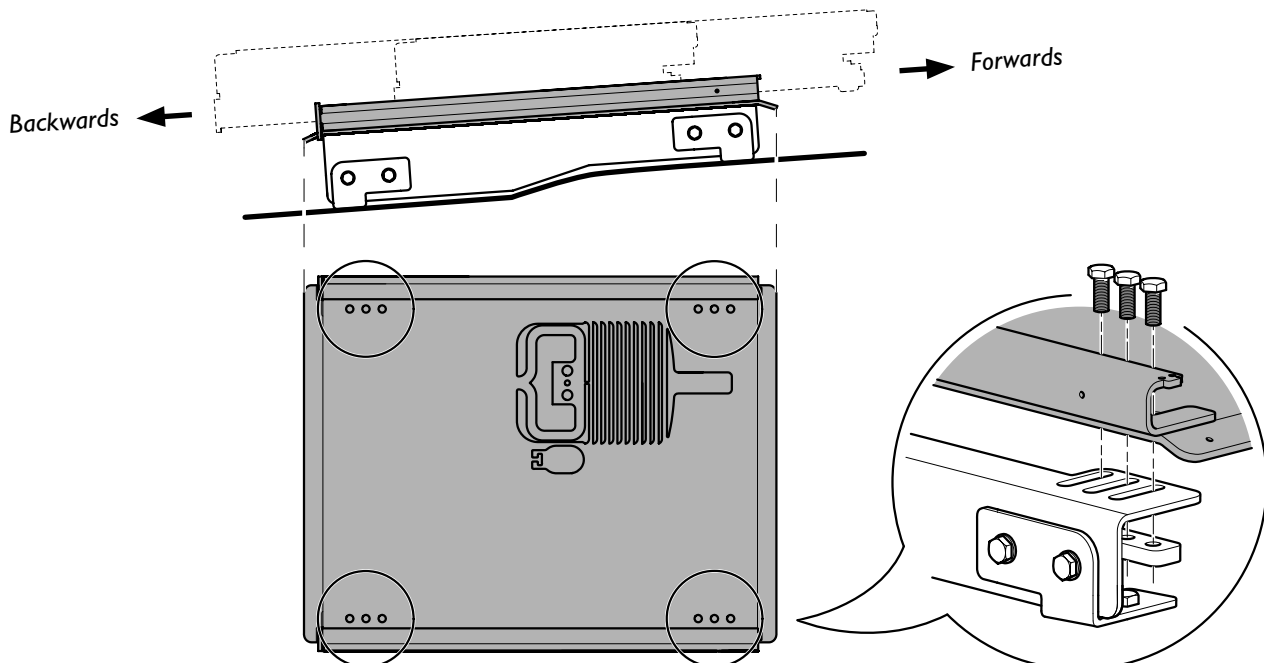Run the unit backwards and forwards to access the attachment points on the base plate. Install Turny Low Vehicle using the nuts and bolts supplied with the bracket.

6. Program new movement paths

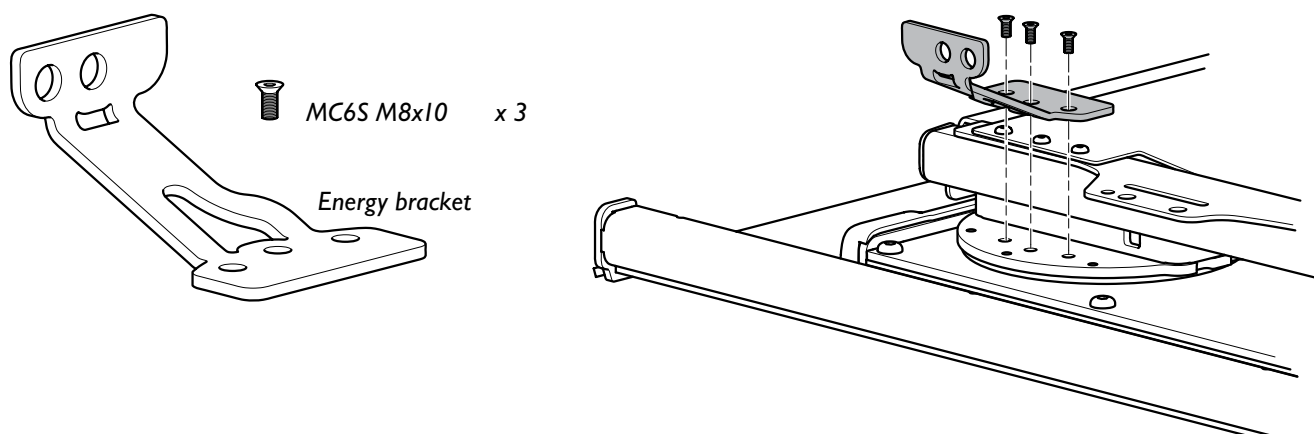
Upon delivery, Turny Low Vehicle uses the default program. In order to facilitate access to attachment points during installation, the unit needs to be reprogrammed, see Chapter II. Note that final programming is not conducted until the installation is completed, and then either from scratch or by editing the existing program.

7. Install mount for lower seat belt anchoring point

Attach the lower anchoring point for the seat belt to the original attachment if possible. In cases where this is not possible, use the mount supplied with the unit and attach it to Turny Low Vehicle as illustrated below.

The mount is best installed before the unit is swivelled out to attach the seat in the next step.

8. Fit seat

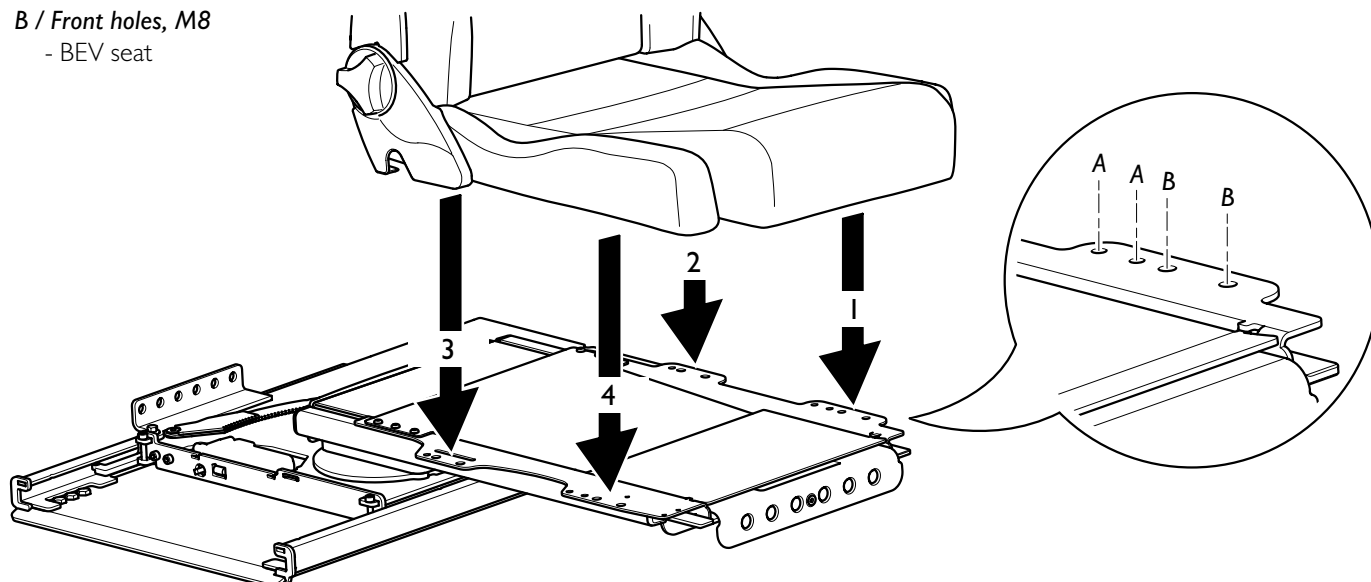
Swivel and extend Turny Low Vehicle a little to gain access to the attachment points for the seat and seat belt. Install the seat in the numerical order illustrated using the nuts and bolts supplied. Start from the attachment point marked with the numeral 1 closest to the door. Note that the BEV seat can be installed in two different positions.

A / Rear holes, M6

- Recaro seat

B / Front holes, M8

- BEV seat

9. Fit seat belt

Always fit the seat belt buckle to Turny Low Vehicle. To avoid the risk of the seat belt injuring the user or being pulled off in a collision, the seat belt buckle must move freely and not press against the user's hip. The seat belt webbing should not be in contact with Turny Low Vehicle.

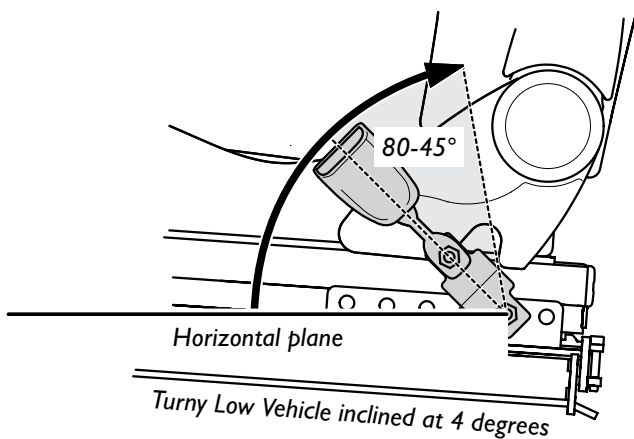
Attach the lower anchoring point for the seat belt to the original attachment if possible. In cases where this is not possible, use the mount supplied with the unit and attach it to Turny Low Vehicle.


Always strive to achieve the following seat belt geometry as illustrated below. Specified degrees are obtained from the legal requirement /

ECE R-14, Revision 5, seat belt attachment points

Belt buckle

Lower anchoring point

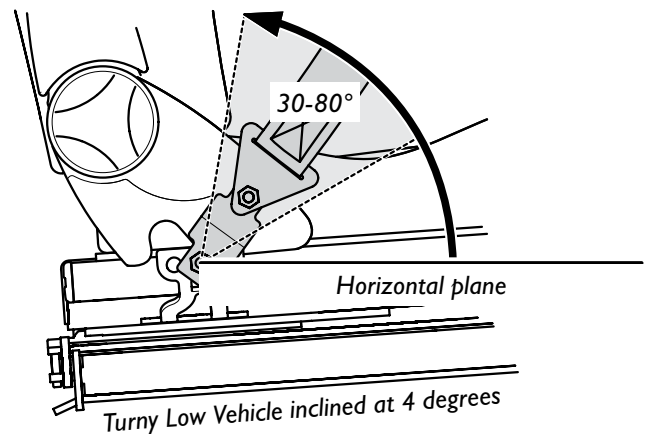If the original belt has any kind of belt pretensioner or switch, adapt the cabling to allow connection to Turny Low Vehicle. The illustration on the next page shows a right-swivelling Turny Low Vehicle.

9.1 Cut original connectors

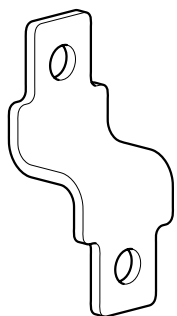
Cut original connectors on belt attachment, belt buckle and associated cabling on the floor under the seat.

9.2 Plug in new connections

Replace original connections according to the illustration on the next page with supplied plug connections. Make sure the cable colours on the plug connections match those of the cable on Turny Low Vehicle.

9.3 Fit seat belt


Fit according to the illustration on the next page using the energy brackets, nuts and bolts supplied. Use the washer supplied if the hole pattern in the OEM belt attachment is too big for the bolt.

Energy bracket x 2

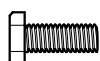The attachment bolts for the energy brackets must be installed from the inside as illustrated below in order to ensure Turny Low Vehicle crash safety.

7/16 UNF x 22 x 4

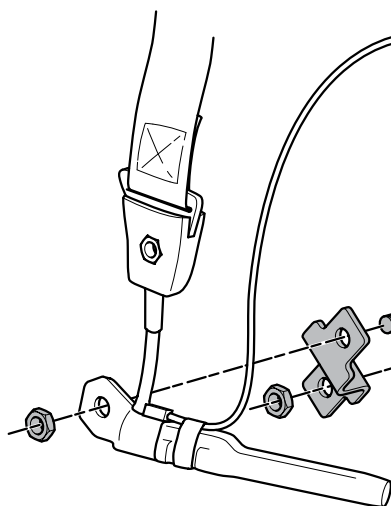
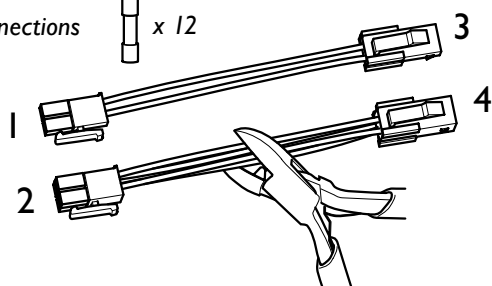
7/16 UNF x 4


12.5x26x2 x 2 (if necessary)

Plug connections x 12


E.g. OEM seat belt attachment with belt tensioner.

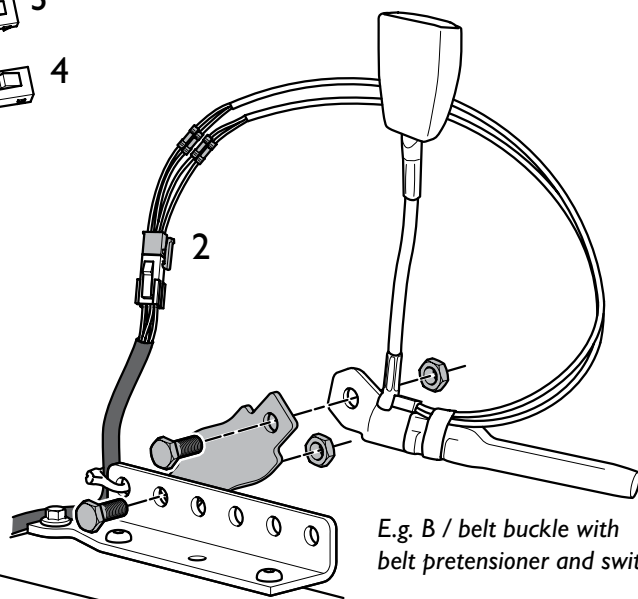Free the cable located under the cover plate in the front centre part of the swivel. Pull the cable through the square hole and attach it to the energy bracket.

E.g. B / belt buckle with belt pretensioner and switch.

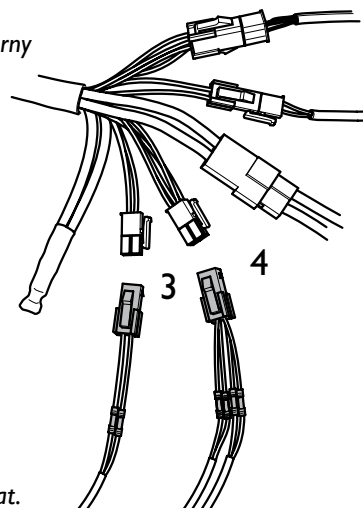Cabling from Turny Low Vehicle.

E.g. OEM cabling under seat.

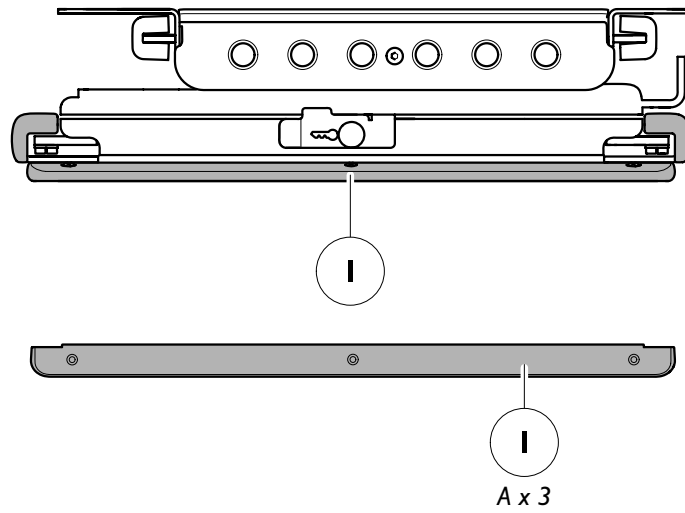
10. Install covers

Note! Swivel Turny Low Vehicle fully into the vehicle. Move the unit backwards until the attachment points for both covers (edge strips) for the base plate below are exposed.


MODEL RIGHT

A  M4 x 8

Front cover/edge strip


Rear cover/edge strip

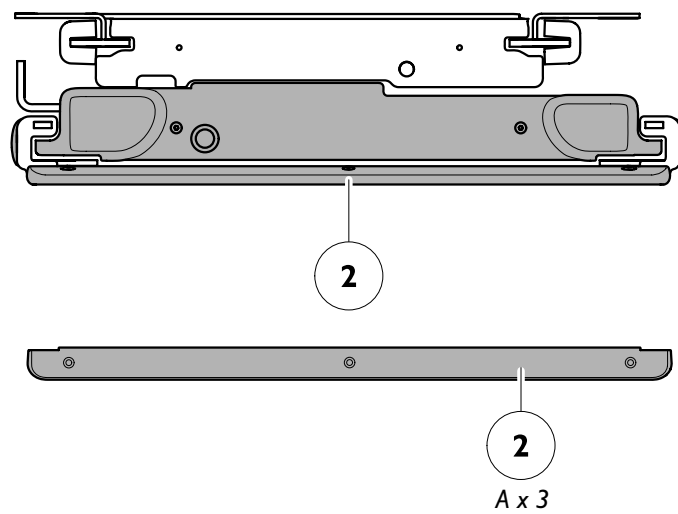
MODEL LEFT

A  M4 x 8

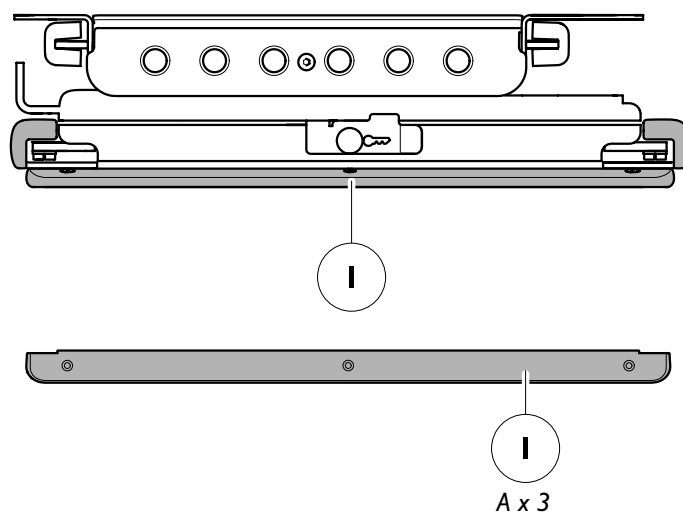Front cover/edge strip



Rear cover/edge strip

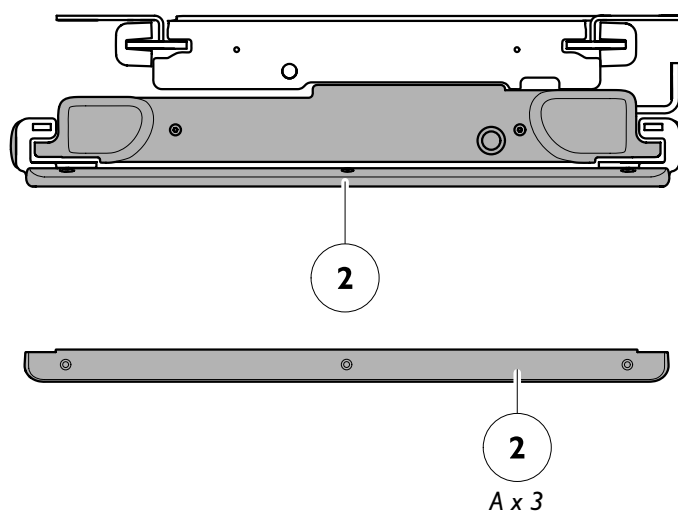II. Program Turny Low Vehicle

II.1 Activate the programming menu

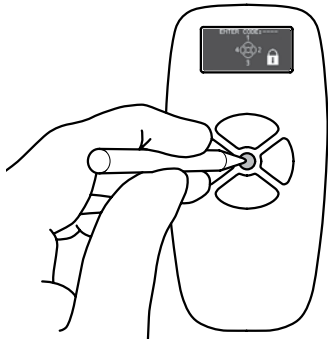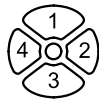Hold down the yellow button until the **Enter Code** screen is shown. Then enter the code 1, 2, 3, 4

When programming, all the lights will flash RED to indicate that Turny Low Vehicle is not following its preprogrammed path and that certain safety functions are disconnected.

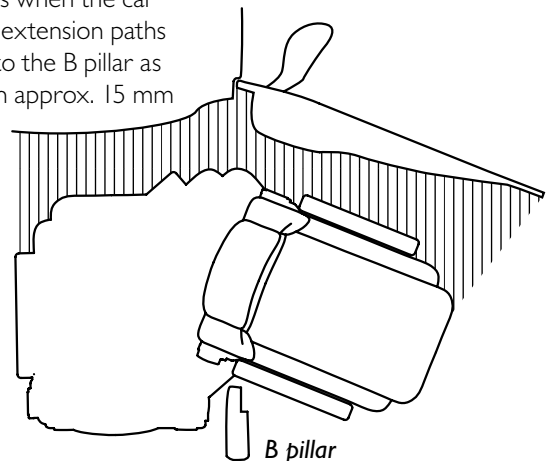
II.2 Program movement paths

Turny Low Vehicle has three motors, one each for lengthwise, swivel and extension. These motors must be programmed to create two separate movement paths for the seat, a **Comfort** and a **Swivel Out**.

The movement path **Comfort** is the lengthwise path that the seat follows when the car door is closed. The **swivel out** movement path combines the swivel and extension paths the seat follows when the car door is open. Program the path as close to the B pillar as possible. The programmed path can vary by +/- 5 mm. Program with an approx. 15 mm margin to prevent damage to the product and car.


For many reasons it is a good idea to program as few operations as possible. Use **Stepless Swivel Out** first to achieve a smooth, speedy movement path.

When first programming it can be difficult to achieve the optimal path. To edit a path, save it and exit programming mode. Run the unit back to its home position and then run it to the point from which you wish to edit the path. Enter programming mode, make the desired changes and save.


Program movement path, comfort

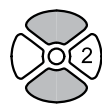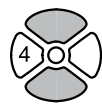1

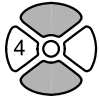
Press 2 for **forwards** until screen 2 comes up.

2

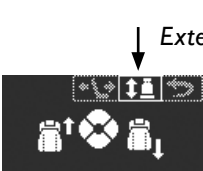
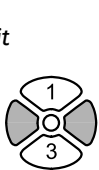
Press 4 for **Backwards** until screen 3 comes up.

- 3   Confirm the rear starting point.
Press 4 for **Backwards** until screen 4 comes up.
- 4   Correct exit position.
Use 2 and 4, **Forwards** and **Backwards**, to run to the desired front end position.
(When screen 5 comes up it is possible to save the end position)
- 5   Press 3 to **Save** and continue programming.
(Screen 6 comes up)
- 6   Use 2 and 4, **Forwards** and **Backwards**, to run to the desired rear end position.
(When screen 7 comes up it is possible to save the end position)
- 7   **Tips!**
The rear end position for comfort is the starting point for **Swivel Out** below. Mark this starting position with a piece of tape. This marking can be useful in the event of any editing of the **Swivel Out** path.
Press 3 to **Save** and continue programming. *(Screen 8 comes up)*

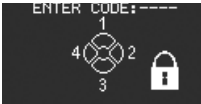
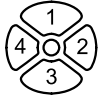
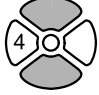
Program movement path, swivel out

Switch between screen 8 and 9 to program this path.

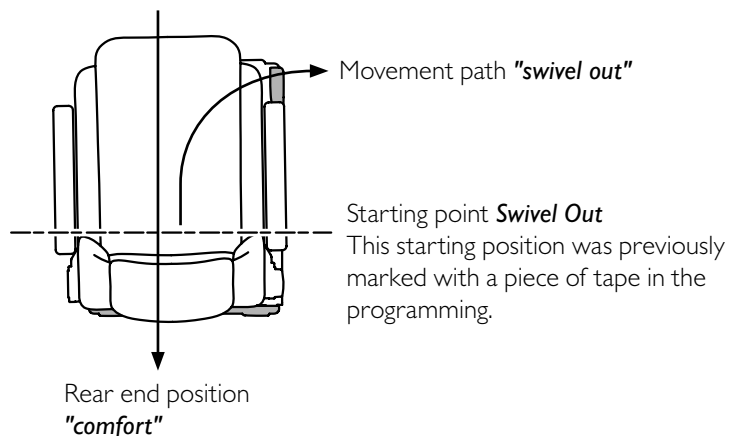
- The figure in the left corner shows how many degrees the seat is swivelled out.*
- 8 **Stepless Swivel Out**   Press 1 to select in the main menu at the top.
Press 2 to activate selected direction (displayed as a large symbol to the right of the screen).
Press 3 for **"Sequentially Swivel Out"** (+1° at a time).
Press 4 to select direction **Gradually Swivel Out**, **Forwards** or **Backwards**.
 - Forwards**  
 - Backwards**  
 - 9 **Extend**   Press 1 or 3 to select the main menu at the top.
Press 2 for **In**.
Press 4 for **Out**.
 - 10 **Exit**   Press 1 to select in the main menu at the top.
Press 3 to save and exit from the programming mode. Turny Low Vehicle is now back in normal operating mode.

Edit movement path, comfort


The front end position allows you to edit and save separately, steps 11-15. To edit and save the rear end position, steps 16-17, editing of the front end position is required first.

- 11**   Run the seat in normal operating mode and swivel the seat fully into the car. Activate the programming menu Hold down the yellow button in the middle of the hand-held control until the **enter code** screen is shown. Then enter the code 1, 2, 3, 4 (**Screen 12 comes up**)
- 12**   Press 2 for **forwards** until screen 13 comes up.
- 13**   Press 4 for **Backwards** until screen 14 comes up.
- 14**   Use 2 and 4, "**forwards**" and "**backwards**", to run to the desired front end position. (**When screen 15 comes up it is possible to save the new end position**)
- 15**   **Alternative A** - Press 1 to "**save**" and exit from the programming mode. Turny Low Vehicle is now back in normal operating mode.
Alternative A - Press 3 to "**save and continue**" and program rear end position. (**Screen 16 comes up**)
- 16**   Use 2 and 4, **Forwards** and **Backwards**, to run to the desired rear end position. (**When screen 17 comes up it is possible to save the new end position**)


The rear end position must be programmed behind the starting position for **Swivel Out**. If the end position is programmed in front of the starting point, it will not be possible to manoeuvre the seat and the programming must be done again.

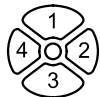- 17**   Press 1 to **Save** and exit from the programming mode. Turny Low Vehicle is now back in normal operating mode.

Edit movement path, swivel out

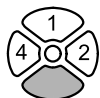It is possible to begin editing from any point on the path. If the first part of the programming is good you do not need to program the whole path. Swivel out in normal operating mode to the point from which new programming is preferred to be done. Activate the programming menu Hold down the yellow button in the middle of the hand-held control until the **enter code** screen is shown. Then enter the code 1, 2, 3, 4


Press 2 for **Continue**. Then program the new swivel and extender, see screens 8-10. Press 1 to undo and cancel.

Press 4 to go straight into the emergency operation menu.

12. Update the software

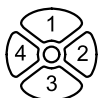A new hand-held control with the new program version is required to update the software in the unit. Open the car door where Turny Low Vehicle is installed and make sure that the hand-held control lights up. Replace the existing hand-held control with the new one. The door will be open until step 3 below.


Activate the programming menu. Hold down the yellow button in the middle of the hand-held control until the **enter code** screen is shown. Then enter the code 4, 3, 2, 1. (**Screen 2 comes up**)

Press 2 to undo and cancel.
Press 4 to commence updating.

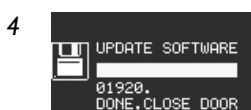
Updating in progress.


Updating is finished
Close the door and open again. (**Screen 5 comes up**)

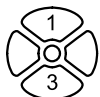If Turny Low Vehicle is installed on the right side of the car, movement path programming may begin; see Chapter 11. If Turny Low Vehicle is installed on the left side of the car, inform the system of this; see next step below.

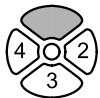Hold down the yellow button in the middle of the hand-held control until the **enter code** screen is shown. Then enter the code 1, 1, 1, 3 (**Screen 7 comes up**)

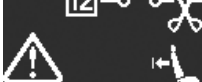
Press 4 for **"continue"** installation.
Press 3 for **"OK"**. (**Screen 8 comes up**)

Start programming of movement paths, see Chapter 11. Note that the choice of installation **"left"** is only saved if programming is done in direct connection to the choice. If this is not done in direct connection you must go in and select the **"left"** installation again.

13. Troubleshooter

Warnings/error messages

	Warning, Incomplete seat swivel-in. Turny Low Vehicle is not in a crash-safe position. The symbol is laterally inverse on a left-side Turny Low Vehicle.
	Low battery charge.
	1. Cable break in the spiral connector to the hand held control. 2. Communication problem between the control card and the hand-held control.
	1. Swivel motor blocked. 2. Swivel motor pulse generator error.
	1. Extender motor blocked 2. Extender motor pulse generator error
	1. Lengthwise adjustment motor blocked. 2. Lengthwise adjustment motor pulse generator error.
	Swivel motor pinch protection activated.
	Extender motor pinch protection activated.
	Lengthwise adjustment motor pinch protection activated.
	Cable break or limit switch for the swivel motor out of service.
	Cable break or limit switch for extender motor out of operation.
	Cable break or limit switch for length adjustment motor out of operation.

Recommended action

Swivel the seat all the way in.

Charge the vehicle battery.

Replace the cable.

1. Check that there are no obstructions.
2. Check the connector to ensure that no cables are loose.
3. Follow the emergency operation instructions (*User Manual*).

If obstructed, Turny Low Vehicle will stop and back a little. Remove the obstruction and try again.

Follow the emergency operation instructions (*User Manual*).

Warnings/error messages

Swivel motor overloaded.

Extender motor overload

Lengthwise adjustment motor overloaded

Swivel-in motor switch activated in wrong position.

Extender adjustment motor switch activated in wrong position.

Lengthwise adjustment motor switch activated in wrong position.

Turny Low Vehicle has lost its position.

Recommended action

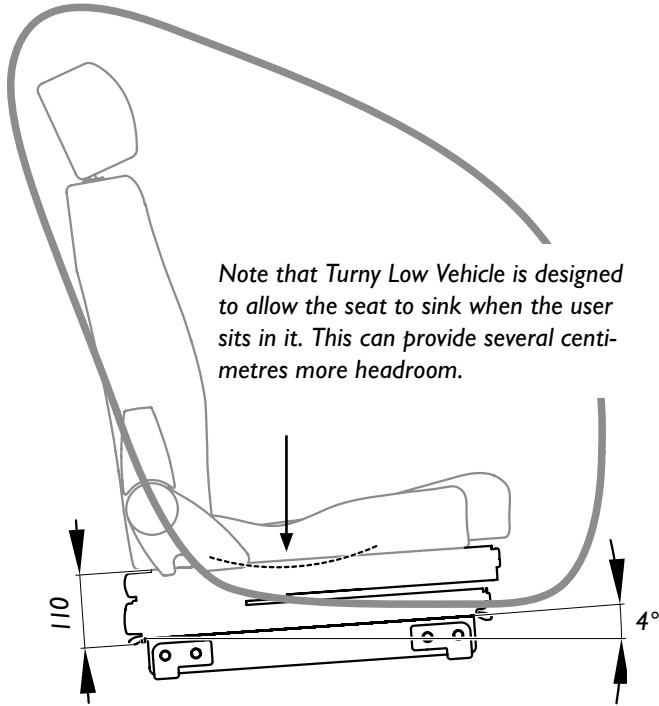
Make sure there are no obstructions and try again with an empty seat.

Follow the **Emergency Operation with Hand Control** instructions to reset Turny Low Vehicle's starting position (**User Manual**).


Follow the **Emergency Operation with Hand Control** instructions to reset Turny Low Vehicle's starting position (**User Manual**).

Technical specification


From the side, fully swivelled in, with the door opening as reference.

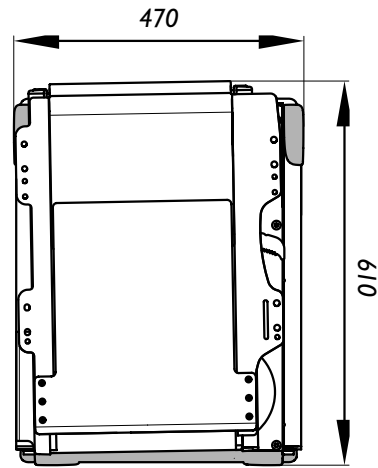From above, fully swivelled in. Note that the specified length and width apply when the top plate is offset 1°.

From above, swivelled out to the right

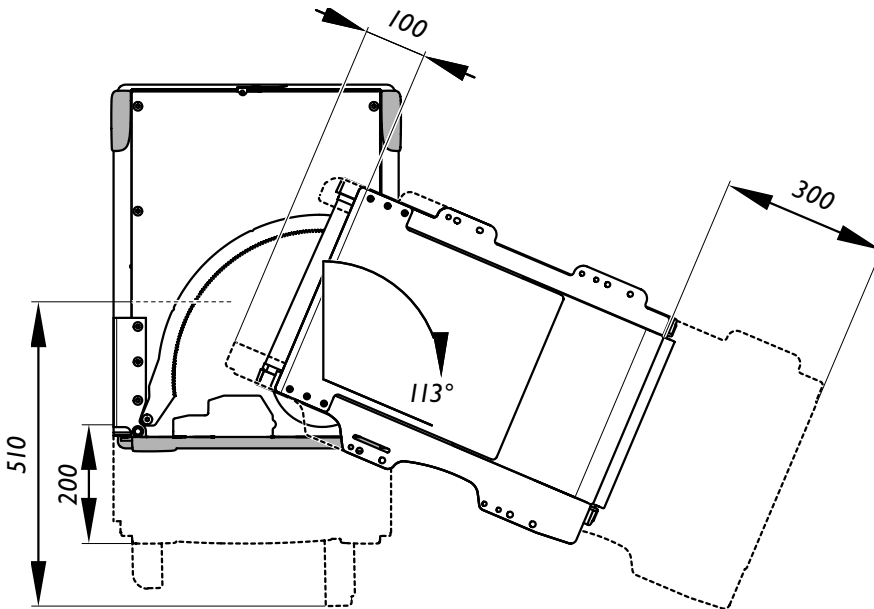Product data

Length.....	610 mm
Width.....	470 mm
Height.....	110 mm
Incline.....	4°
Product weight, without seat and accessories.....	65 kg
Operating voltage.....	12V
Fuse.....	15A
Power consumption in sleep mode, door switch required	0.8mA

Product performance

Lengthwise movement in programming mode.....	510 mm
Lengthwise movement in comfort mode.....	max 200 mm
Swivel angle.....	113°
Extension forward.....	300 mm
Extension backwards, ReXtension™	100 mm
Weight capacity *.....	140 kg

* *Weight capacity refers to the total weight of the seat, its accessories and the user.*

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Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005
Quality system certified in accordance with ISO 9001:2008

Illustrations, descriptions and specifications in the user manual are based on current product information.
Autoadapt reserves the right to make alterations without previous notice.

© 2015 Autoadapt AB

www.autoadapt.com

facebook.com/autoadapt
twitter.com/autoadapt

AUTOADAPT AB

Åkerivägen 7
S-443 61 Stenkullen
Sweden
Phone: +46 302 254 00
E-mail: info@autoadapt.com

Worldwide Safe Mobility Solutions

Included in our corporate group are Autoadapt,
BraunAbility, Bruno, Unwin and Foca-Braun.