

Carospeed® Classic

User manual

Thank you for choosing a

Carospeed® Classic from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit www.braunability.eu.

Once again, thank you for placing your confidence in our products!



Carospeed® Classic may only be used by individuals who have a prescription to do so.



Do not use Carospeed® Classic as a handle to get inside the car.



Contact an authorized technician if the Carospeed® Classic appears to be wiggly sideways.



Carospeed® Classic shall be inspected by an authorized technician if a car crash has occurred.


Hill-holder

The hill-holder function enables the user to brake the car at the same time as the gear shift is set. Activate the hill-holder by pressing the button on the left hand side of the handle. It can only be activated while braking the car, when sufficient brake power is achieved and ignition is on.


The hill-holder lets go automatically as soon as the handle is pushed forward (braking).




The hill-holder must not be used as a parking brake.

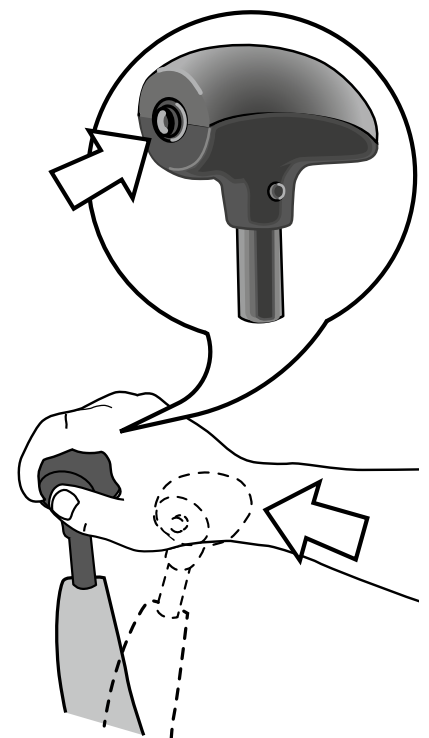

Start and drive off

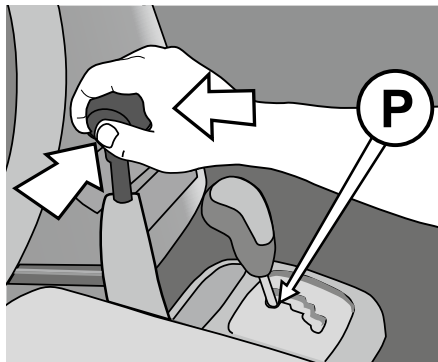


1 Start the engine with the gear lever in position P. Engage the hill-holder. While the hill-holder is engaged, gear positions D or R can be selected without the car starting to roll.

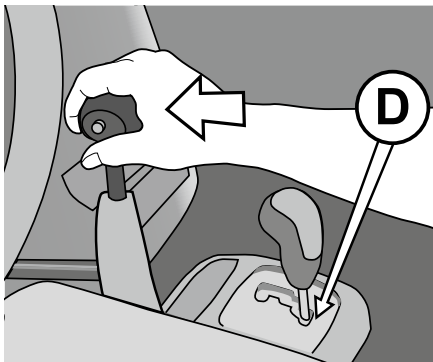

2 Release the parking brake. Un-activate the hill-holder (press the handle forward and the hill-holder lets go automatically). Now you can start driving.

To brake, push the handle forward. To accelerate, pull the handle backwards.

Park the car

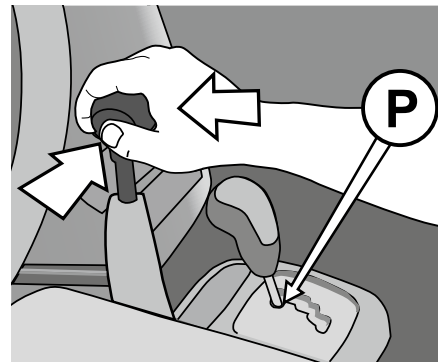








1 Brake the car and activate the hill-holder as soon as the car has stopped. Set the gear shift to parking position (P) and pull the parking brake. Un-activate the hill-holder by pushing the handle forward.

EN
User manual

Primary electrical functions

Connections are made according to individual customer needs. The installer is responsible for marking the connections.

- 1. Wiper 
- 2. Wash and wipe 
- 3. Flashers, left/right 
- 4. Headlight 
- 5. Horn 
- 6. Hill-holder 
- 7. Cruise 
- 8. Tiptronic +
- 9. Tiptronic -

