Caroseat® Flex

Adaptable car seat

SV	4	Bruksanvisning
EN	10	User manual
DE	16	Gebrauchsanweisung
FR	22	Instructions d'utilisation
ES	28	Instrucciones de uso
IT	34	Istruzioni per l'uso

SV

EN

DE

FR

ES

IT

Svenska	4	
Säkerhet och skötsel	5	
Handhavande	6	
English	10	
Safety and care	11	
Operation	12	
Deutsch	16	
Sicherheit und Pflege	17	
Handhabung	18	
Français	22	
Sécurité	23	
Utilestation	24	
Espagnol	28	
Seguridad	29	
Uso	30	
Italien	34	
Sicurezza	35	
Uso	36	
<i>Garanti/Warranty/Garantie/ Garantía/Garanzia</i>	40	

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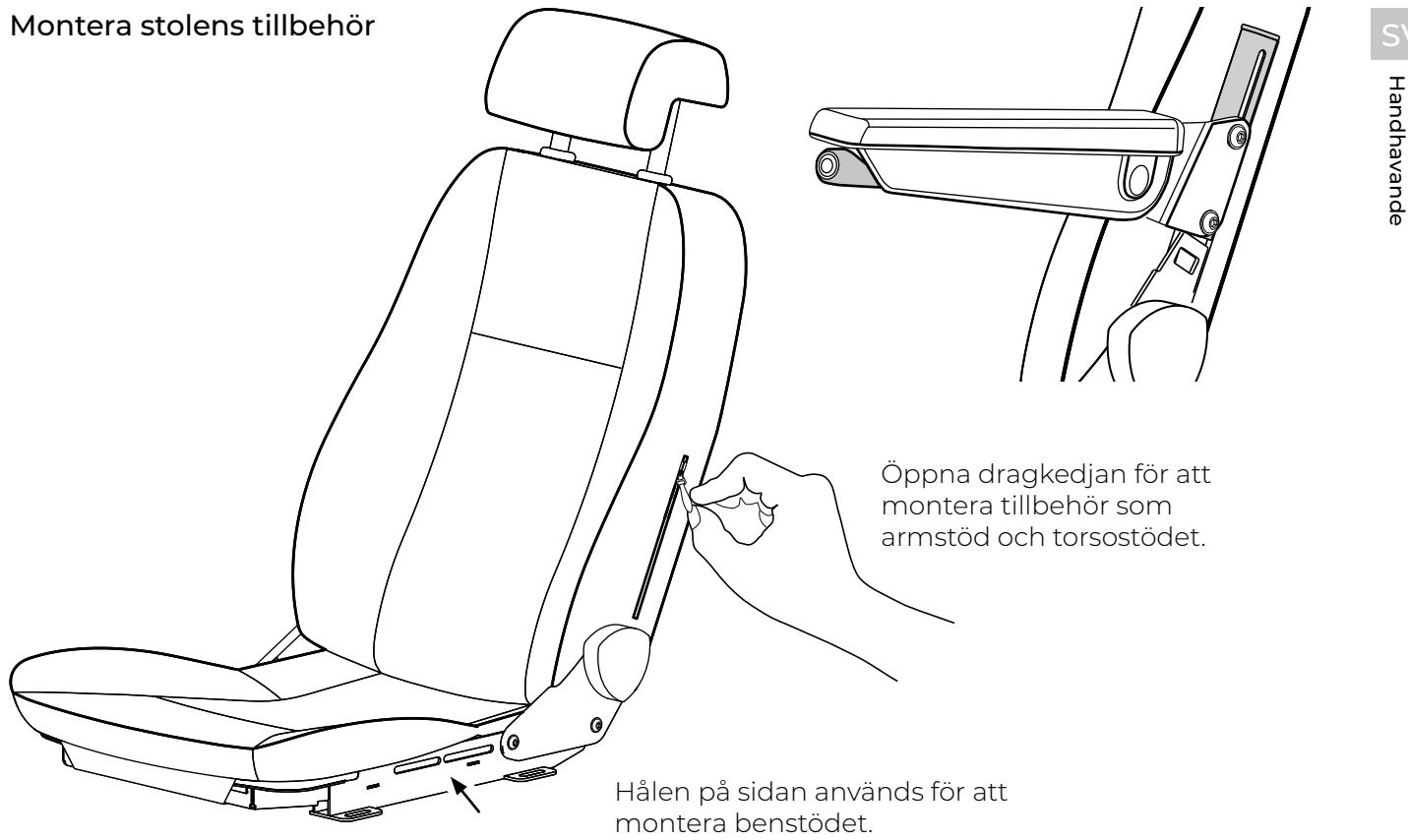
Thank you for choosing a

Caroseat® Flex from BraunAbility

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. You should carefully read and follow the warnings and notices provided in this manual, and keep the manual in a safe place.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit www.braunability.eu.

Once again, thank you for placing your confidence in our products!

BraunAbility Safety Centre
Accredited test laboratory
www.braunability.eu

Indicates a warning for a situation in which harm, body injury or death could result if the warning is ignored.

Indicates a notice that provides interesting or helpful information.

Safety and care instructions

Regularly inspect the product for damage, wear or malfunction. If any problems are identified replace components immediately.

If the vehicle is involved in an accident when any vehicle safety systems (e.g. airbags) are deployed, remove the product from service and replace immediately. If in doubt please contact BraunAbility.

EN

Safety and care instructions

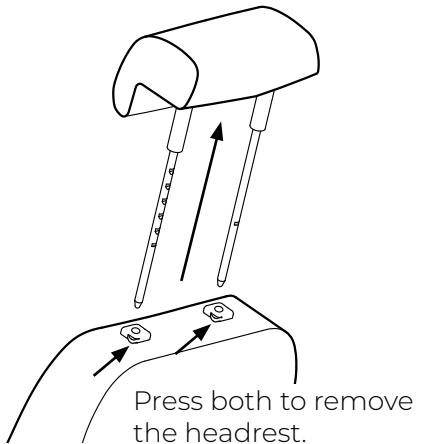
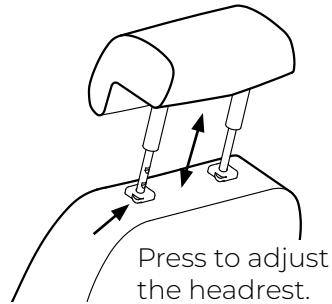
To clean the Caroseat® Flex use warm soapy water and a clean soft cloth. Rinse with clear water and allow to air dry. To disinfect, use mild spray disinfectant and do not use products containing bleach.

When cleaning or disinfecting, do not immerse or flood the Caroseat® Flex in the disinfectant or water.


Operation

Backrest and headrest

Operation

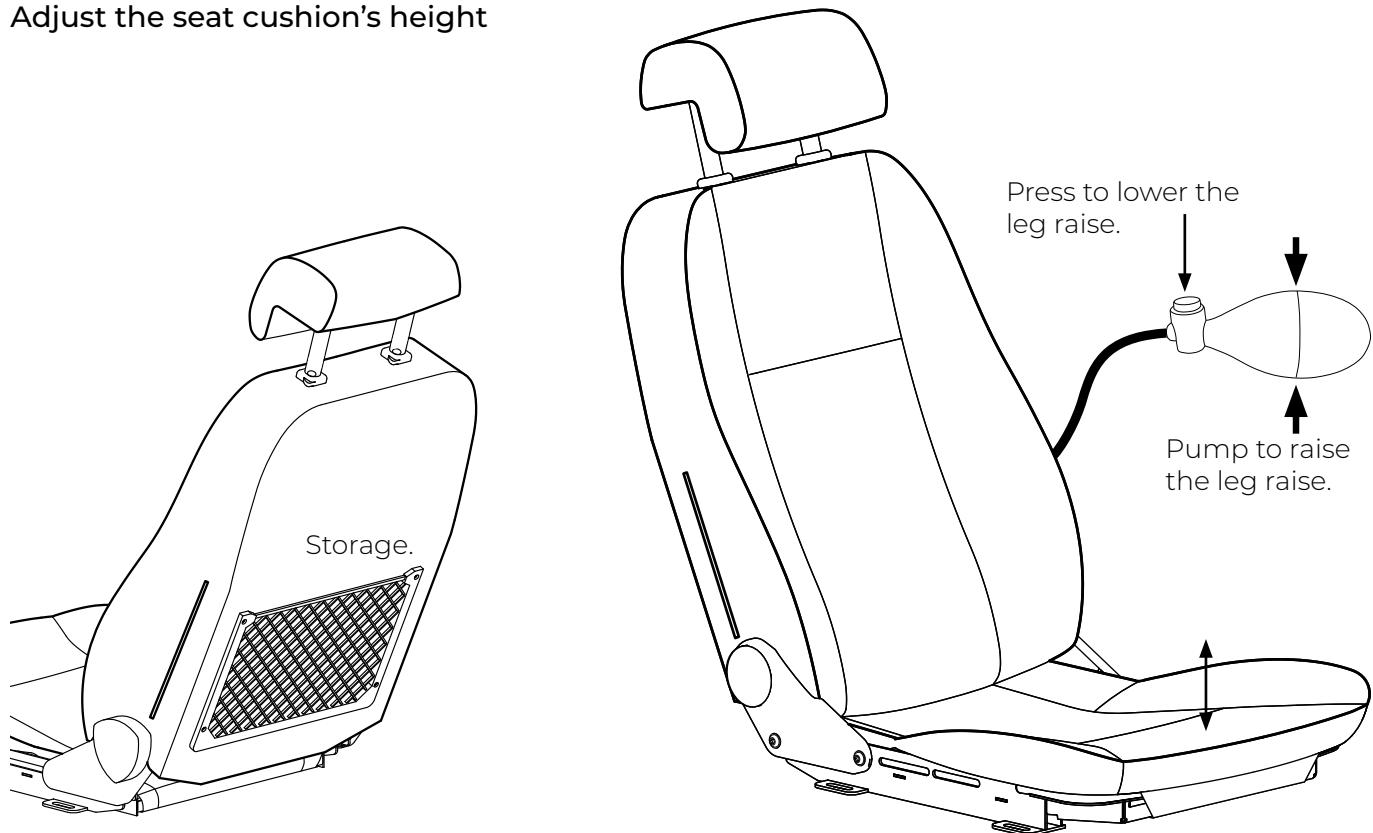Adjust the seat cushion's height


EN


Operation

Operation

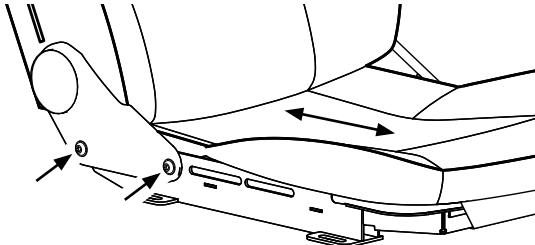
Adjust the seat length

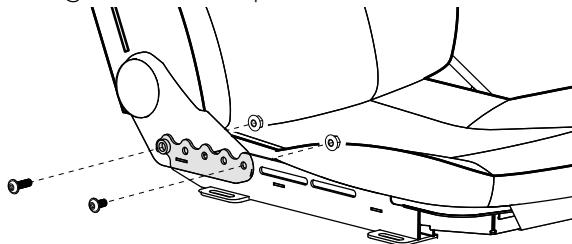
Remove the screws on both sides of the seat to change the seat depth.

Longest seat depth possible.

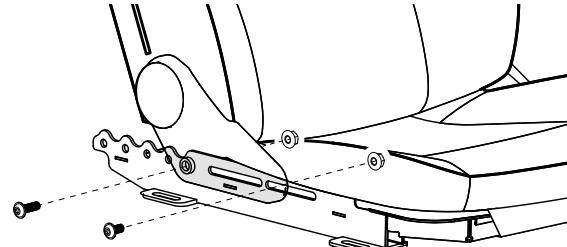Shortest seat depth possible.

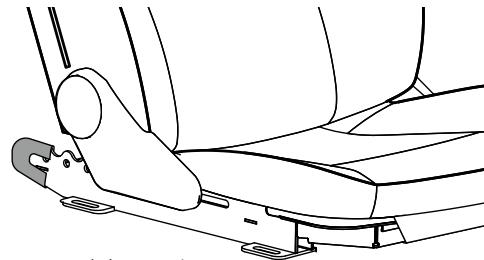Fit the two rubber trims.

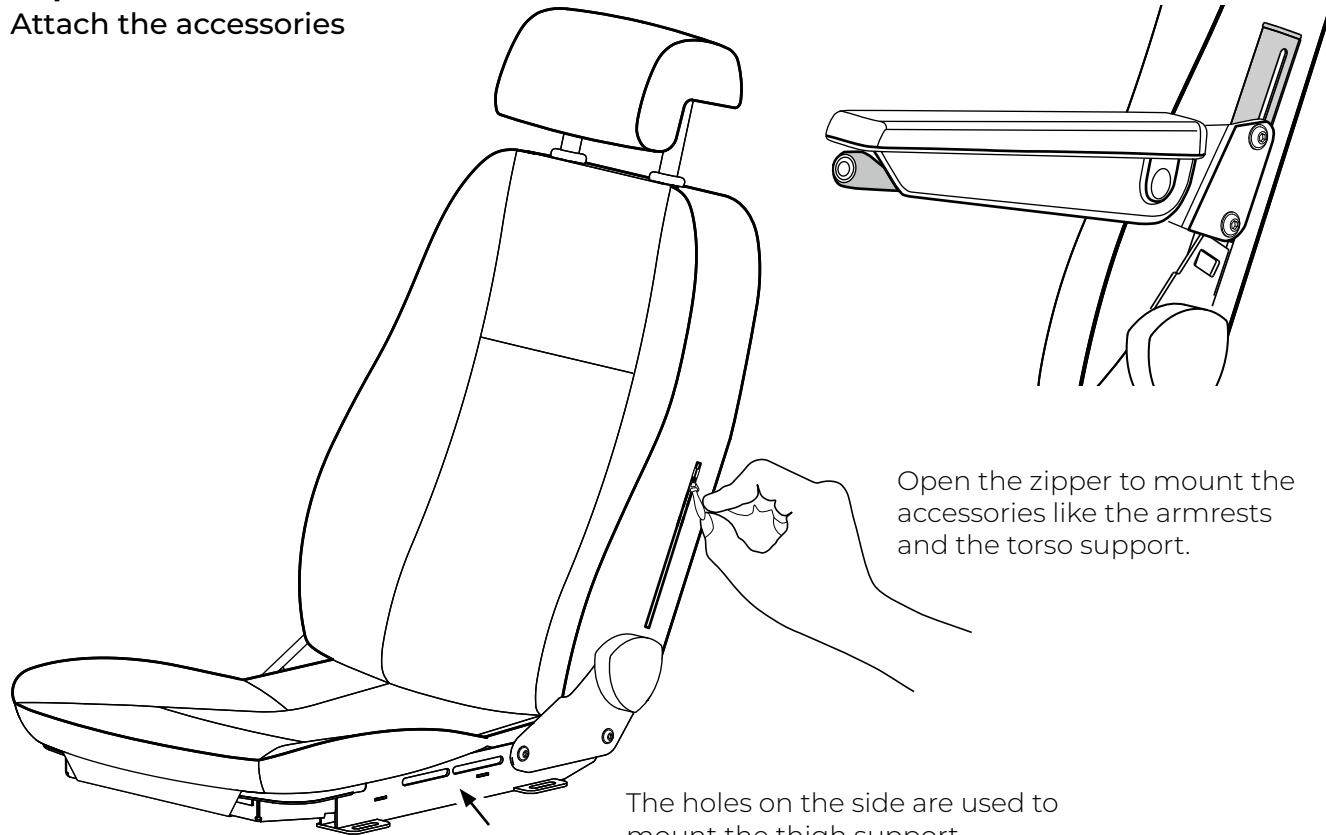
Always use a round hole for one of the two screws on each side.

When Caroseat Flex is used with a shortened seat depth, the two rubber trims supplied must be fitted.

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BraunAbility products are extensively tested and our full warranty applies to BraunAbility equipment when used as instructed. For further details on specific applications please contact the Customer Service Centre.

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Accred. no. 1976
Testing
ISO/IEC 17025

BraunAbility Europe AB, Stenkullen, Sweden is accredited by SWEDAC as testing laboratory in accordance with ECE R14, ECE R17 and ECE R145.

BraunAbility Europe AB is certified in accordance with ISO 9001:2015 and ISO 14001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.
BraunAbility Europe AB reserves the right to make alterations without previous notice.

© 2024 BraunAbility Europe AB

BraunAbility Europe AB
Åkerivägen 7
443 61 Stenkullen
Sweden

Phone: +46 302 254 00
E-mail: info@braunability.eu
www.braunability.eu