


How to set up the BraunAbility Remote app

1. Download the BraunAbility Remote app from Google Play or App Store.
2. Activate Bluetooth on your smart-phone.
3. Press the Teach-in button (A) quickly three times in a row and follow the setup wizard in the app.
4. If the Teach-in button on the receiver isn't accessible, run the lift until it is.

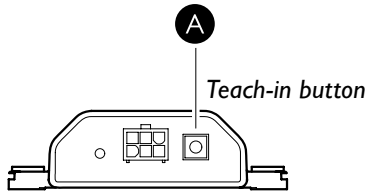


Before using this device, download and read the user manual available at: www.braunability.eu

BraunAbility Europe AB
Åkerivägen 7
443 61 Stenkullen
Sweden

Phone: +46 302 254 00
E-mail: info@braunability.eu
www.braunability.eu


427359-3 2020/12

 **BraunAbility®**

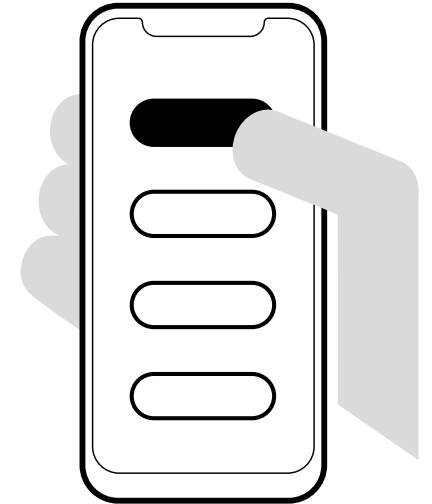

BRAUNABILITY REMOTE

Quick guide for Global lift

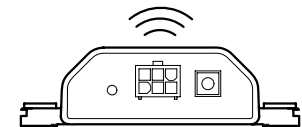

How to install the BraunAbility remote receiver

1. Choose placement, either *inside, on* or *near* the pump module. Ensure that the **Teach-in button** on the receiver is easy to access. Mount the receiver in step 4.

2. Connect and colour match the supplied cable adapter. Insert the spade terminals as follows:

Green – Remove the female spade terminal from the Valve (A) and insert the Green wire from the cable adapter.

Yellow, White, Black and Brown - Separate the male/female spade terminals (B) and insert the cable adapter.

Red - Remove the female spade from the Switch (C) and insert the Red wire from the cable adapter.

3. Connect the adapter cable to the receiver.
4. Mount the receiver according to the illustration using the supplied adhesive pads. In some cases it might be necessary to drill (**5 mm**) and use the supplied clips.



Be careful. Check for obstructions to ensure free space before drilling.

5. Use the supplied cable tie to tidy up the excess cable.

