


How to set up the BraunAbility Remote app

1. Download the BraunAbility Remote app from Google Play or App Store.
2. Activate Bluetooth on your smartphone.
3. Run the Turny Evo to a position where the Teach-in button (A) on the receiver is accessible. You have to lift the curtain to access the button.
4. Press the teach-in button x3 and follow the setup wizard in the app.

Turny Evo from above, swiveled out to the right.

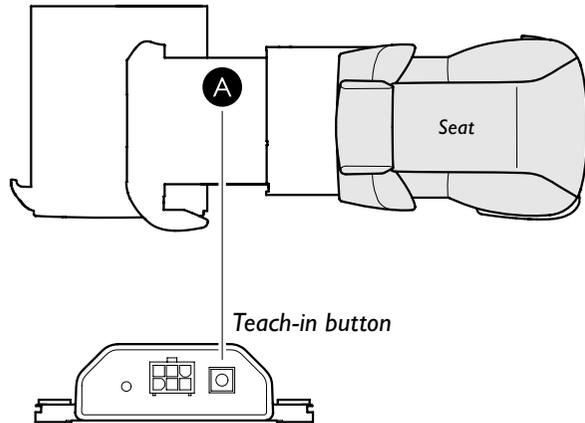

Before using this device, download and read the user manual at one of the following websites:

www.braunability.com
www.braunability.eu

BraunAbility
631 West 11th Street
Winamac, IN 46996
USA.


Phone: +1 574 946 6153
Fax: +1 574 946 4670
www.braunability.com

BraunAbility Europe AB
Åkerivägen 7
443 61 Stenkullen
Sweden

Phone: +46 302 254 00
E-mail: info@braunability.eu
www.braunability.eu


508020 Rev. A / 427362-1 2019/04

 **BraunAbility®**

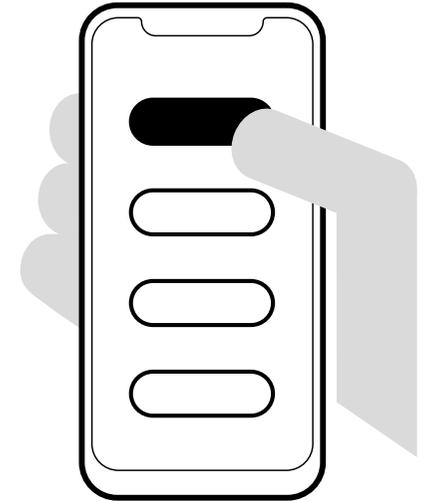

BRAUNABILITY REMOTE

Quick guide for Turny Evo

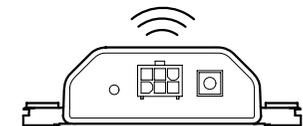

How to install the BraunAbility remote receiver

1. Swivel and extend the Turny Evo as far out as possible.
2. Unscrew the curtain retaining screw and feed curtain back in its roll. This will keep it from getting caught during the rest of the installation.
3. Connect the receiver to the socket (A) on the Turny Evo. Align the receiver with the holes at the position (B). Fasten using the supplied screws and spring washers.
4. Carefully route the cables and secure them underneath the swivel motor using the cable tie. Ensure that no cables are pinched when the unit is in motion.
5. Pull out the curtain and reattach with the screw.

