


How to set up the BraunAbility Remote app

1. Download the BraunAbility Remote app from Google Play or App Store.
2. Activate Bluetooth on your smartphone.
3. Press the teach-in button x3 and follow the setup wizard in the app.

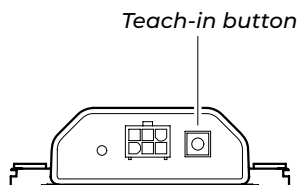


Before using this device, download and read the user manual available at: www.braunability.eu

BraunAbility Europe AB
Åkerivägen 7
443 61 Stenkullen
Sweden


Phone: +46 302 254 00
E-mail: info@braunability.eu
www.braunability.eu

427445-1 2022/03

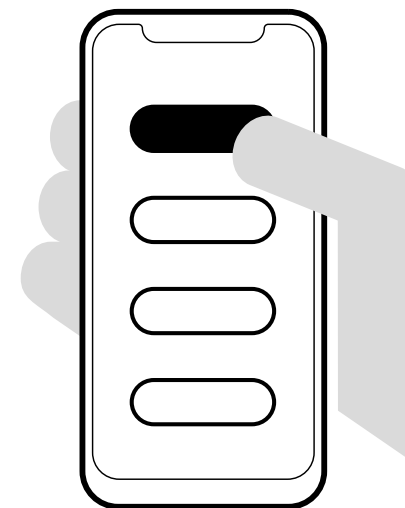

BRAUNABILITY REMOTE

Quick guide for Carolift 100

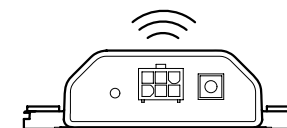

How to install the BraunAbility remote receiver

1. Choose placement, near the Carolift control board. Ensure that the **Teach-in button** on the receiver is easy to access. Mount the receiver in step 3.
2. Connect the patch cable between Carolift and the splitter.
3. Connect the adapter cable between the splitter and the receiver.
4. Connect the hand control to the splitter on the same side as the adapter cable.
5. Mount the receiver according to the illustration using the supplied adhesive pads. In some cases it might be necessary to drill (**5 mm**) and use the supplied clips.



Be careful. Check for obstructions to ensure free space before drilling.

